




POLYMED
MEDICAL DEVICES

MICROFUSION SET

1 Unit

MICRO INFUSION SET WITH AIRVENT

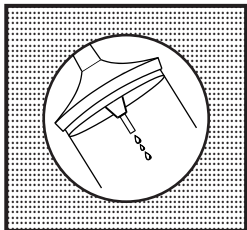

DIRECTIONS FOR USE :

1. Close the Regulating Clamp.
2. Remove the protective cap from the spike and insert the spike into the I.V. fluid container to its full length.
3. Open the airvent cap of piercing spike.
4. Squeeze the drip chamber and adjust the fluid level till half filled.
5. Open regulating clamp and allow solution to pass until air bubbles in the tube are removed completely, close the regulating clamp.
6. Connect the IV set with I.V. device and regulate the flow by adjusting regulating clamp to achieve desired flow rate.

CAUTION :

- Do not use if package is damaged.
- For single use only.
- Rubber part to be used only for injecting the medicine.
- Discard after single use.
- Do not use for blood transfusion.
- Do not use if protective caps are loose or missing

For Feedback : customercare@polymedicure.com

“Gravity feed only”

•NON-TOXIC •NON-PYROGENIC

60 DROPS = 1±0.1 ml



EC REP Obelis S.a
Boulevard Général Wahis 53
1030 Brussels, BELGIUM
Tel : +(32) 2. 732.59.54
E-mail : mail@obelis.net

REF :

Needle Size :

LOT :

MFG. :

EXP. :


Code. No. :

POLY MEDICURE LTD.

Plot No.104-105, Sector-59, HSIIDC Indl. Area,
Faridabad-121004, (Haryana) INDIA.

Customer Care No. : +91-129-4287053

Rev.10-19/05

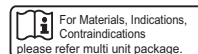

For Materials, Indications,
Contraindications
please refer multi unit package.

DOC NO. PML/U-00777	Date : 31-12-05	Dgn. By	Checked By	Approved by
Soft blister Paper pouch for Micro with Airvent (Export)	Size :200x100mm			
Colour Break <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/> Green	Specifications : 1. 3. 2. 4.	(Designer)	(Q.A. Person)	(Q.A. Incharge)
Rev.	Ammdment	Issue No. 11	Scale : 1:1	Sheet 1 of 1
8	ifu has been changed and added pore symbols		Remarks	Date of Amdt.
9	Changed CE No. 2460 instead of CE No. 0434			20-2-16
10	Polymed Logo Changed			24-09-17
				22-5-19