




POLYMED
MEDICAL DEVICES

ONCOFUSION SET

IV infusion set

with 0.2 Micron in line filter and priming filter
(with airvent)

- DEHP free
- 0.2 Micron Filter
- Latex free


20 DROPS = 1±0.1 ML 

DIRECTIONS FOR USE

1. Close the Regulating Clamp.
2. Remove the spike protector cap and insert the spike into the IV fluid container. Gently squeeze and release drip chamber.
3. Open the airvent cap of spike.
4. Keep the 0.2µ filter in vertical position.
5. Open the regulating clamp.
6. Priming takes place automatically up to integrated filter (maintains a close system thus prevents contamination). Close the regulating clamp.
7. Remove the priming filter cap and connect the IV set with IV device and start infusion and adjust the flow rate with the help of regulating clamp.

CAUTIONS

- Do not use if protective cap is loose or missing.
- Do not use if package is opened or damaged.
- Close the airvent during periods of interrupted infusion therapy.
- For gravity feed only.
- Use Y-site only for injecting the medicines using aseptic techniques.
- Priming filter for single use only.
- The product should be used only by qualified healthcare professionals.
- Do not re-sterilize. Discard the set after single use.
- Reuse and cleaning of product may alter their structural and mechanical properties. It may lead to infection or other illness/injury.
- The product should be replaced and disposed of as per facility approved protocol or CDC guidelines.
- Store at room temperature, avoid excessive heat, protect from direct sunlight and moisture.
- Poly Medicure Limited will not be responsible for any direct incidental or consequential damages resulting from reuse of product.
- Do not use for blood transfusion & blood component

 For Materials, Indications, Contraindications please refer multi unit package.

Approx. Priming Volume : 23ml (Approx.)

● Non-Toxic ● Non-Pyrogenic

EC/REP Obelis S.a
Boulevard Général Wahis 53
1030 Brussels, BELGIUM
Tel : +(32) 2. 732.59.54
E-mail : mail@obelis.net

REF :

LOT :

 Mfg. :

 Exp. :

Code. No. :

STERILE EO



LATEX FREE



POLY MEDICURE LTD.

Plot No.104-105, Sector-59, HSIDC Indl. Area,
Faridabad-121004, (Haryana) INDIA.
Customer Care No. : (0129) 4287053



CE
2460

1 Unit

Rev. 08-19/05

For Feedback : customercare@polymedicure.com

DOC NO. PML/U-00838	Date :09-06-08	Dgn. By	Checked By	Approved by
Oncofusion Set DEHP FREE (For Export)	Size : 100x235mm	(Designer)	(Q.A.Person)	(Q.A. Incharge)
Colour Break <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/> BLUE	Specifications : 1. 3. 2. 4.	Issue No. 09	Scale : 1:1	Sheet 1 of 1
Rev.	Amendment	Remarks		Date of Amdt.
(4)	Delete the barcode for make common for all type			22-3-14
(5)	IFU CHANGED, Deleted Needle size, Generic name revised			02-02-15
(6)	Changed IFU section and added caution instead of warning and added pore symbol			11-06-16
(7)	Changed CE No. 2460 instead of CE No. 0434			25-09-17
(8)	Polymed Logo changed			24-05-19