



POLYVOL BURETTE SET

MEASURED VOLUME FLUID ADMINISTRATION SET

1 Unit

DIRECTIONS FOR USE :

1. Close the upper clamp & regulating clamp (lower clamp).
2. Remove anti kink device before using the product from bottom.
3. Remove the spike cover and insert it into the I.V. fluid container to its full length.
4. Open the upper clamp and air vent to allow about 25 ml solution into the graduated burette. Close the upper clamp.
5. Squeeze the drip chamber and adjust the fluid level till 1/3rd filled.
6. Remove the cap of the connector of IV set and slightly open regulating clamp to remove air from the tubing. Close the regulating clamp.
7. Open upper clamp to allow solution flow into graduated chamber until the desired volume is obtained. Do not fill the chamber beyond its maximum capacity. Close upper clamp and connect the set with IV device.
8. Open the regulating clamp. Adjust drop rate and control infusion with the regulating clamp. Allow the infusion to continue.
9. The valve will shut off the fluid flow when solution level in graduated chamber is zero.
10. When more solution is required, close the regulating clamp and open upper clamp to fill graduated chamber until desired level. Close upper clamp and squeeze the drip chamber a little to release the float valve.



Salient features :

- Shut-off type floating valve.
- Drop size Approx. 60 Drops/ml. "GRAVITY FEED ONLY"

POLYMED
MEDICAL DEVICES

POLY MEDICURE LTD.

Plot No.104-105, Sector 59, HSIIDC Indl. Area,
Faridabad-121004, (Haryana) INDIA.
Customer Care No. : +91-129- 4287053
For Feedback : customercare@polymedicure.com

Caution :

- Use only if package is intact.
- Do not use if protective caps are loose or missing.

CE 2460



STERILE EO

- NON-TOXIC
- NON-PYROGENIC

Obelis S.a
Boulevard Général Wahis 53
1030 Brussels, BELGIUM
Tel : +(32) 2. 732.59.54
E-mail : mail@obelis.net

REF :

Burette size :

LOT :

Mfg. :

Exp. :

Code. No. :



For Materials, Indications,
Contraindications
please refer multi unit package.

Rev. 02-19/05

DOC NO. PML/U-01515		Date : 10-10-11		Dgn. By	Checked By	Approved by
Soft blister Pouch for Polyvol Export common for 110ml and 100ml		Size : 100x355mm				
Colour Break		Specifications :		Issue No. 03	Scale : 1:1	Sheet 1 of 1
 BLUE		1.	3.			
		2.	4.			
Rev.	Ammendment				Remarks	Date of Amdt.
(1)	Die size changed for 110ml and 100ml					30-10-18
(2)	Polymed logo changed					25-5-19
(3)						
(4)						