



PHOTOFUSION SET

1 UNIT

DEHP Free Vented I.V. Infusion Set (Protects against Photolysis)

DIRECTIONS FOR USE :

1. Close the Regulating Clamp.
2. Remove the protective cap from the spike and insert the spike into the I.V. fluid container to its full length. Squeeze the drip chamber and adjust the fluid level till half filled.
3. Open the air vent cap of piercing spike.
4. Open the regulating clamp, priming takes place automatically up to integrated filter (maintains a close system thus prevents contamination). Close the regulating clamp.
5. Remove the priming filter cap.
6. Connect the IV set with I.V. device and regulate the flow by adjusting regulating clamp to achieve desired flow rate.



POLYMED
MEDICAL DEVICES

For Feedback : customercare@polymedicure.com

NON-TOXIC, NON-PYROGENIC

CAUTIONS :

- Do not use if packing is damaged
- Do not use for blood transfusion
- Discard the set after single use
- Do not use if protective caps are loose or missing

"GRAVITY FEED ONLY"

For Materials, Indications & Contraindications please refer multi unit package.



Obelis S.a
Boulevard Général Wahis 53
1030 Brussels, BELGIUM
Tel : +(32) 2. 732.59.54
E-mail : mail@obelis.net



STERILE EO



20 DROPS = 1±0.1 ML

REF :

Approx. Priming

Volume :

LOT :

Mfg. :

Exp. :

Code No. :

POLY MEDICURE LTD.

Plot No. 104-105, Sector-59, HSIIDC Indl. Area,
Faridabad-121004, Haryana, INDIA
Customer Care No. : +91-129-4287053

Rev. 01-19/05

Doc. No. PML/U-01693	Date : 14-05-19	Design by	Checked by	Approved by
Unit Pack Design of Photofusion Set For Exoprt	Size : 235x100mm P. Area :	Designer	QA Person	QA Incharge
Color Break <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/> Blue	Specifications : 1. 2.			
Rev.	Amendment		Remarks	Date of Amdt.
1.	Polymed Logo Changed			24-5-19
2.				
3.				
4.				