
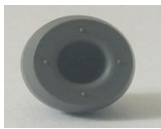



HEBEI FIRST RUBBER MEDICAL TECHNOLOGY CO., LTD

Description	PET tube	Cap Color	Reference	Size (mm)	Liquid Capacity (ml)	Additive	Shlef-life (month)
Set of PET tube for vacuum blood collection are included: PET tubes, Rubber stoppers and Plastic caps in detail as followed							
<i>Pet tuber for vacuum blood collection</i>			12. 5*75	12. 5*75	5	No	36
<i>Rubber stoppers for vacuum blood collection tubes</i>			14-F	13mm			
<i>Plastic cap vacuum blood collection</i>		Blue	627#	13mm			

Instruction for use

1. Samples should be mixed by inverting gently 5 times immediatellym after collection.
2. Store blood to clot for 20 minutes in vertical position and centrifuge it within 1-2 hours.
3. Separate the serum of the sample to another container right after the centrifugation.

HEBEI FIRST RUBBER MEDICAL TECHNOLOGY CO., LTD
Address: NO. 518, CHENGUANG ROAD, ZHENGDING TECH. INDUSTRIAL PARK, SHIJIAZHUANG CITY, HEBEI PROVINCE, CHINA.

Tel: +86-13313016063

Fax: +86-311-89197808

Website: www.xiangyi.co