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* Some kits shown may not be available in all countries.

** The value encoded in the instrument settings is around 87 mL for **cobas c** 701/702 analyzers and 88 mL for the **cobas c** 513 analyzer to account for the dead volume of the bottles.

English**System information**

ECO-D: ACN 946

Intended use

EcoTergent is an additive to the reaction bath to reduce surface tension on **cobas c** systems.

Summary

EcoTergent is added to the reaction bath. It acts as a surfactant to minimize the formation of bubbles that could potentially interfere with the photometer readings.

Reagents - working solutions

Detergent

Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents.

Disposal of all waste material should be in accordance with local guidelines. Safety data sheet available for professional user on request.

For USA: Caution: Federal law restricts this device to sale by or on the order of a physician.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:

Danger

H314 Causes severe skin burns and eye damage.

H410 Very toxic to aquatic life with long lasting effects.

Prevention:

P273 Avoid release to the environment.

P280 Wear protective gloves/ protective clothing/ eye protection/ face protection.

Response:

P303 + P361 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water.

P304 + P340 IF INHALED: Remove person to fresh air and keep comfortable for breathing. Immediately call a POISON CENTER/ doctor.

P305 + P351 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/ doctor.

P391 Collect spillage.

Product safety labeling follows EU GHS guidance.

Contact phone: all countries: +49-621-7590, USA: 1-800-428-2336

Reagent handling

Ready for use

Storage and stability

Store protected from light.

Shelf life at 15-25 °C: See expiration date on **cobas c** pack label.

On-board in use and refrigerated on the analyzer: 12 days

cobas c 701/702 analyzers:

On-board on the Reagent Manager: 1 hour

Materials provided

See "Reagents – working solutions" section for reagents.

Materials required (but not provided)

See "Order information" section

Assay

Use EcoTergent as specified in the respective instructions for use.

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard (for USA: see <https://usdiagnostics.roche.com> for definition of symbols used):

CONTENT

Contents of kit

Volume after reconstitution or mixing

GTIN

Global Trade Item Number

**FOR US CUSTOMERS ONLY: LIMITED WARRANTY**

Roche Diagnostics warrants that this product will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

COBAS and COBAS C are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2018, Roche Diagnostics

Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

Distribution in USA by:
Roche Diagnostics, Indianapolis, IN
US Customer Technical Support 1-800-428-2336