MÔ TẢ SẢN PHẨM:

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**PRODUCT DESCRIPTION**

SAKOSKIN SEA is a medical device

solution mist dispersed evenly over the nasal mucosa help hygiene, cleaning, antibacterial, help prevent rhinitis, sinusitis, help nasal ventilation, breathable, restore natural moisture to the nose.

COMPOSITION:

Natri clorid 0.90%, Benzalkonium clorid 0.01%, Water (RO) 99.09%.

SPECIFICATION:

Box 1 bottle capacity: 70ml

INTENDED USE :

- The solution nasal spray mist is used for children over 3 months of age, pregnant women and adults.
- nasal spray solution has antibacterial effects, help prevent rhinitis, sinusitis.
- To help clear away mucus, nose, helps nasal ventilation, breathable, naturally restore moisture to the nose.

DIRECTIONS FOR USE:

- Open the protective cover vertically.
- Slightly lean back, put a hose on each side of the nose, hand print definitive 2-3 times per nostril.
- Use 3 to 4 times / day.
- Clean and dry the nozzle, protective cap after use.

STORAGE:

Store the dry cool place, out of reach of children.

CONTRAINDICATIONS:

Hypersensitivity to any component of the product.

IMPORTANT WARNINGS AND PRECAUTIONS:

Do not use the alternative solution when using nasal hygiene in every bottle.

Specifications: Manufacturer**Number of Medical Equipment:**

Manufactured in: DOLEXP HAR

Address: Lot C-2 Dai An Industrial Zone Expanding Town Lai Cach, Cam Giang District, Hai Duong Province.