**1. Product Description**

**bt products** Hematology Cleaner is a isotonic detergent-based cleaning reagent, intended to remove cellular debris and proteins from the fluid pathways of automated and semi-automated hematology analyzers. It is suitable for in vitro diagnostic use.

**bt products** hematology cleaner can be used in, BT-PRO21 and BT-PRO2401 hematology analyzers. Hematology Cleaner must be used with appropriate **bt products** Brand Hematology Lyse, Hematology Diluent and other required operational reagents for best results. **bt products** Hematology Cleaner contains bacteriostatic and fungistatic agents as preservatives. All compounds are biodegradable.

**2. Reagent Composition**

Sodium chloride	< 0.8 %
Sodium phosphate	< 0.3 %
Brij 35	< 0.2 %
Antimicrobials	< 0.2 %

**Physical form:** Slight blue liquid.  
**pH:** 7.0

**Handling Precautions:** Avoid contact with eyes, skin and clothing. Wear laboratory gloves when handling the product. The product may be harmful if ingested or inhaled. Keep the bottle closed when not in use.

**3. Reagent Storage**

**Storage conditions:** Stored at 8 to 30°C.

**Shelf Life:** See package labelling for expiration date.  
**Discard open containers:** 60 days after opening.

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Bilimsel Tıbbi Ürünler Paz.San.Tic.Ltd.Şti.