180mm

120mm

180mm

120mm

80mm

**PURIFY®**

**N95-4ply**  
Filtering Facepiece

20 PCS

With earloops and shapeable nosepiece

Adjustable nose fit

Flat-folded for face-fit

Comfortable earloops

Chin-fit coverage

120mm

**INSTRUCTION**

- Step 1:** Unfold the mask. Place the inner side on your mouth and nose with the nosepiece up.
- Step 2:** Wear the 2 straps around your ears. Position the respirator so as to comfortably cover your nose and mouth.
- Step 3:** Use your fingers from both hands to shape the nosepiece.
- Step 4:** Check the mask Seal Fit by placing both hands over the mask and exhale. If air leaks around the nose, check the nosepiece. If air leaks around the edges, check if the respirator fits if your face.

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**APPLICATION**

**N95-4ply respirator is designed for:**

- Protects healthcare workers against various aerosol-borne particles.
- Protects patients during handling or working with patients, animals, swarming, swarming, swarming rodents and other dirty equipment.
- Used to help to protect from the transfer of, shingles, body fluids, and other particulate materials.

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**WARNINGS**

- DO NOT REUSE
- Dispose appropriately after usage
- This mask does not provide oxygen nor does it filter oil/moist particles. Do not use for painting, spray of handling gas, smoke, or vapour. Do not use for highly corrosive and environment.
- People with respiratory issues should consult health professionals before wearing

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Manufacturer/Hersteller

**KIM DUC GROUP JOINT STOCK COMPANY**

Head Office: Unit 27/04, Sao Gon Trade Center,  
27 Tonkin, Thang Di, District 1, Ho Chi Minh City, Vietnam

Factory: KIM DUC GROUP JOINT STOCK COMPANY - THANG AN BRANCH  
Lot B1.1 - B1.3, Road V11, Vinh Loc 2 Industrial Zone,  
Long Hiep Commune, Ben Luc District, Long An Province, Vietnam.

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