80mm

180mm

120mm

180mm

120mm

INSTRUCTION

1. Unfold the mask. Place the inner side on your ears and nose with the nosepiece up.
2. Wear the 2 straps around your ears. Position the respirator so as to comfortably cover your nose and mouth.
3. Use your fingertips from both hands to shape the nosepiece.
4. Check the Mask Seal Fit by placing both hands over the mask and exhale. If air leaks around the nose, check the nosepiece. If air leaks around the edges, check if the respirator has fit your face.

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With earloops and shapeable nosepiece

20 PCS

PURIFY®
Khẩu trang N95-5 lớp

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Che phủ khuôn cằm

20 Cái/hộp

APPLICATION

N95-5ply respirator is designed for:

- Provide reliable protection against certain aerosol-borne particles.
- Provide protection during handling or working with powders, dusts, smogging, fogging, or solid materials and other dusty operations.
- Used daily to protect from the transfer of droplets, body fluids, and other particulate materials.

WARNING

- DO NOT REUSE.
- Dispose appropriately after usage.
- This mask does not provide complete ear relief if filter is blocked partially. Do not use for painting, spray or handling gas, smoke, or vapour. Do not use for highly contagious air environments.
- People with respiratory issues should consult health physicians before wearing.

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