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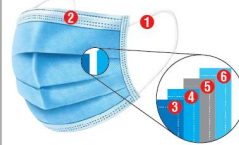
4 Layers - 1 Time use *

Dust and microbial resistant

- ★ Strap.
- ★ Nose clip.
- ★ 2 layers of non-woven dust infiltration fabric.
- ★ A layers of microbial filter.
- ★ A layers of fashionable non-woven fabric.

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Medical Masks 4 Layers - 1 Time use

ECO LIFE MEDICAL FACE MASK helps prevent smoke, dust, air pollution and virus from entering to your body. It can be used efficiently on roads, construction sites, healthcare and industrial environments.

INGREDIENTS:

1. Soft and elastic rubber band.
2. Flexible nasal wire.
3. 2 layers of non-woven dust infiltration fabric.
4. 1 layer of microbial filter.
5. 1 outside layer of fashionable non-woven fabric.

INSTRUCTION:

- Hold the mask by the ear loops.
- Place the loops around your ears.
- Bend or pinch the stiff edge to the shape of your nose.
- Pull the bottom of the mask over your mouth and chin.
- PRESERVATION: Place the mask in dry, dust and high temperature free environment (over 40°C or 100°F).
- WARNING: Only use the mask once.
- Keep away from places of a high risk of catching fire.
- Keep out of children's reach.
- Dispose the mask appropriately.
- Do not use the product if it is torn or not in its original form.
- Expiry date: 5 years after manufactured date.
- Manufactured date: On the package.
- 50 masks per load.
- Size: 175 mm X 95 mm (LxĐ)
- Circulation number:

ĐỨC HUY DEVELOPMENT PRODUCTION LTD.,CO

ADDRESS: AN177 Tran Dai Nghia Street, Tan Nhut Commune, Binh Chanh District, HCM City
TEL: 028.39555.868 - Fax: 028.39555.867 - 028.333.888
Email: khastrangytedechuy@gmail.com
Website: khastrangytedechuy.vn