

BIRTHING BED
NIXIE - Ref.2100



Medipréma
SIMPLIFIED USER MANUAL




Mars 2016-v1.0



BIRTHING BED
NIXIE - Ref.2100



médipréma
SIMPLIFIED USER MANUAL

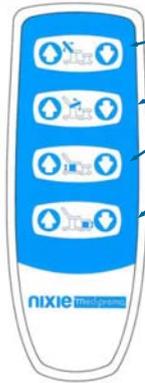

- (1) Back section up and down
- (2) Seat section up and down
- (3) Chair up and down
- (4) Stool up and down (if connected to the chair)

To move up and down the complete bed (chair+stool), please press simultaneously keys 3 and 4

Stirrups handling

- ◆ To change the position of the bracket, pull down the black button to release the locking pin and then pull or push the bracket to the desired position




Stool handling

- ◆ Connected to the chair, the height of the stool is adjustable with the bed remote control.
- ◆ When disconnected, it is adjustable directly with the switch located on the stool, on the front right (if the battery option is existing).
- ◆ There is a tilt available on the stool. You have to action the lever located under the seat, at the back.



Safety precautions of use

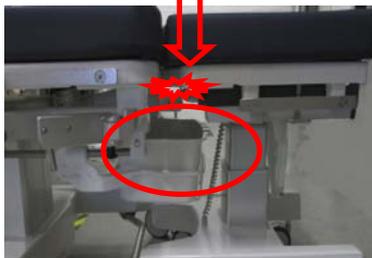

PUSH THE BASIN ALL WAY IN.



DON'T USE STIRRUPS AS FOOTRESTS. It can't withstand the forces. Use only dedicated accessories.



DON'T STORE THE STIRRUPS DIRECTLY ON THE BED. RISK OF BREAK. Stirrups, when not used, must be removed from their bracket and stored outside the bed.

Mars 2016—v1.0



The emission of a beep alerts you on an abnormal situation (failure, default risk). Check the situation (conflict? Overload?) before continuing any bed movement.

Médipréma - ZA Node Park Touraine - 470 rue Gilles de Gennes - BP8 - 37310 TAUXIGNY - FRANCE
Médipréma Tel. +33 (0)2.47.28.47.87 - Fax +33 (0)2 47 27 35 85
 www.mediprema.com - E-mail : info@mediprema.com

