### Mẫu nhãn phụ

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# OPASTER'ANIOS

## Ortho-phthalaldehyde 0.55%

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 LABORATOIRES  
**ANIOS**  
An Ecolab Company

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## EN High level disinfectant for medical devices

**INDICATIONS:** High level disinfectant for medical devices: surgical and medical equipment, endoscopy equipment and heat-sensitive equipment. **COMPOSITION:** Ortho-Phthalaldehyde solution (0.55%) in a buffered medium, colouring, perfume. **MICROBIOLOGICAL PROPERTIES:** Ready to use solution. Bactericide in 5 min, 20°C [EN 1040, EN 13727, EN 14561]. Mycobactericide in 5 min, 20°C [EN 14348, EN 14563]. Fungicide in 5 min, 20°C [EN 1275, EN 13624, EN 14562]. Sporidical activity against Clostridium difficile O27 spores in 30 min, 20°C [EN 14347, EN 13704]. Viricide in 5 min, 20°C [EN 14476]. Active in accordance with EN 14476 in 5 min, 20°C on PRV (surrogate of HBV), BVDV (surrogate of HCV), Vaccinia virus, Herpesvirus. **INSTRUCTIONS FOR USE:** Use without dilution. Clean and rinse thoroughly all instruments before soaking in OPASTER'ANIOS. Cover the basin. Respect the indicated contact time for the required antimicrobial activity. Do not add any product into OPASTER'ANIOS. Rinse thoroughly with sterile or filtered water to eliminate any trace of the product, then dry. To guarantee the efficacy of the disinfectant, check the Ortho-Phthalaldehyde level with a OPASTER'ANIOS testing strip at the start of the session, when restarting after an interruption and at least once a day. Maximum storage of the soaking bath: 14 days. **PRECAUTIONS FOR USE:** Safety data sheet available on request. Contains phthalaldehyde. May produce an allergic reaction. Use in well-ventilated areas. Wear protective gloves. If necessary, call a POISON CONTROL CENTER or a physician. Keep out of reach of children. Do not reuse the packaging. Dispose of unused product and its container as hazardous waste. Reserved for professional use. Class IIb medical device. Made in France.

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