

2

M-30R RINSE

【PRODUCT NAME】

M-30R RINSE

【MODEL】

M-30R

【PACKAGE SPECIFICATIONS】

5.5L、5.5L×2、20L

【INTENDED USE】

For use on MINDRAY BC-3200/BC-3600 hematology analyzer. For use as an azide-free, filtered isotonic solution for cleaning hematology analyzer. Use as directed in instrument operation manual.

【ACTIVE INGREDIENTS】

Sodium Sulfate Anhydrous.....	3.0-5.5 g/L
Sodium Chloride.....	7.5-11.5 g/L
Anti-fungal and Anti-bacterial Agents.....	1.0-3.0 g/L
Polyoxyethylene Ester.....	0.3-1.5 g/L

【STORAGE CONDITION AND SHELF LIFE】

The product must be stored in the temperature of 2°C~30°C(35°F~86°F), The shelf life is 24 Months.

The product must be used in the temperature of 15°C~30°C(59°F~86°F), and it stays valid for 60 days after being opened.

【APPLICABLE ANALYZER MODELS】

The product applies to Mindray BC-3200/BC-3600 Hematology analyzer manufactured by Shenzhen Mindray Bio-Medical Electronics Co., Ltd..

【INSTRUCTIONS FOR USE】

- 1 Keep Hematology Analyzer Reagent between 15°C-30°C during use.
- 2 Refer to Mindray Operation Manual to obtain instrument-specific instructions for appropriate use of Hematology Analyzer Reagent.

【WARNINGS AND PRECAUTIONS】

- If the reagent accidentally gets into your mouth, seek medical assistance immediately.
- If the reagent spills on your skin, wash it off with plenty of water immediately.
- If the reagent spills into your eyes, wash it off with plenty of water immediately.

【NOTICE】






- Do not use expired products.
- If the product has been frozen(either partially or completely), warm it up to room temperature and then mix it by gentle inversion. Verify background results before analyzing patient samples.If background results of the reagent are abnormal due to transportation, keep the reagent still in room temperature for 24 hours before use.
- If reagents are contaminated or become abnormal due to other reasons, stop using and then replace them at once.
- Be sure to dispose of reagents, waste and contaminated packing materials according to government regulations.
- The following conditions may produce interference to test results: expired reagent; reagent contaminated by dusts; unclean sample dilution; reagent mixed with those of other manufacturers; residue reagent mixed with newly opened reagent; reagent not used as required.

【CUSTOMER SERVICE DEPARTMENT】

Manufacturer: Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

Address: Mindray building, Keji 12th Road South, Hi-tech Industrial Park,Nanshan, Shenzhen, 518057 P.R.China

Website: www.mindray.com

				
Batch code	Use-by date	Temperature limit	Manufacturer	In vitro diagnostic medical device

E-mail: service@mindray.com

Address:

Tel: +86 755 81888998

Fax: +86 755 26582680

© 2014 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights Reserved.For this Operator's Manual, the issue date is 2014-02.

4

M-30E E-Z CLEANSER

【PRODUCT NAME】

M-30E E-Z Cleanser

【INTENDED USE】

For use on Mindray BC-3000/BC-3000Plus/BC-3000CT/ BC-3200 hematology analyzer.
For use as a cleaning reagent for cleaning hematology analyzer periodically. Use as directed in instrument operator's manual.

【Active Ingredients】

Proteolytic Enzyme.....	3.0-10g/L
Surfactant.....	0.3-1.5g/L
Sodium Chloride.....	3.0-5.0g/L
Anti-fungal and Anti-bacterial Agents	0.5-2.5g/L
Buffering Agents.....	1.0-4.0g/L

【Warnings and Precautions】

Do not inhale and/or ingest. In case of inhaling or ingestion, seek medical advice immediately.

Avoid contact with skin. In case of contact with skin, rinse immediately with plenty of water.

Avoid contact with eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

【Storage and Usage】

The product should be stored at 2°C to 30°C (35°F to 86°F) and keep container closed.

The product should be used at 15°C to 30°C (59°F to 86°F) once opened, and it is stable for 60 days.

Dispose of waste product, unused product and contaminated packaging in accordance with governmental requirements.

Keep container closed when not in use.

Tel: +86 755 26582888
Fax: +86 755 26582680
www.mindray.com.cn

 Shanghai International Holding Corp. GmbH(Europe)
Eiffestraße 80, 20537 Hamburg Germany

© 2009-2010 Shenzhen Mindray Bio-medical Electronics Co., Ltd. All rights Reserved.
For this Operator's Manual, the issued Date is 2010-05 (Version: 2.0).

