



HIGHTERGENT

CAT. NO. : HIT9703

SIZE : 2,000 ml

PREPARATION AND USE

The resulting solution is a Cleaning Solution for the Chemistry analysers that should be connected via wash line 3.

This product has been designed particularly for the Chemistry analysers and is a phosphate free solution with an antibacterial effect.

STABILITY AND STORAGE

Solution should be stored at +15 to +25°C. Stable as supplied until the expiry date quoted on the bottle. The diluted wash solution is stable for 4 weeks in the absence of contamination when stored at +15°C to +25°C.

SAFETY PRECAUTIONS AND WARNINGS

For *in vitro* diagnostic use only. Do not pipette by mouth. Exercise the normal precautions required for handling laboratory reagents.

Wear protective gloves and safety glasses for handling.

If solution comes into contact with skin or mucous membranes, flush immediately with copious amounts of water, and seek medical advice.

Health and Safety Data Sheets are available on request.

The reagents must be used only for the purpose intended by suitably qualified laboratory personnel, under appropriate laboratory conditions.