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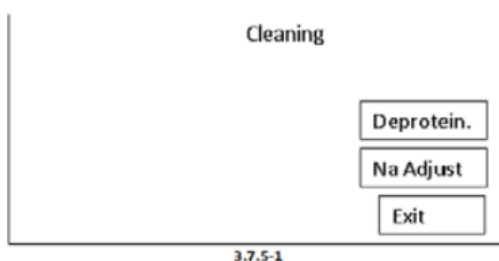
[IVD]

Intended Use

Daily Cleaning Solution is used for automatically performed and manually programmed cleaning procedures on the HumaLyte Plus 3 and HumaLyte Plus 5.

For cleaning procedures please refer to the user manuals for HumaLyte Plus 3 [REF] 17470/10 and HumaLyte Plus 5 [REF] 17470/20

Contents

DAILY Cleaning Solution is filled in plastic vials, closed with white screw caps, labelled

Cat.-no. / Product Name:	17470/71 / Daily Cleaning Solution
Aggregate state	liquid
Appearance	colourless
Density	Approx. 1 g/ml
Smell	None
pH	6.9
Chemical constituents	< 1% Pepsin < 1% Trypsinase < 1% Papain

Preparation

Daily Cleaning Solution is ready for use and can directly be applied on the HumaLyte Plus 3 and HumaLyte Plus 5.

Storage and Stability

Daily Cleaning Solution is stable 24 months from manufactured date and up to the stated expiry date if stored at 5...35°C. After opening the stability is within 12 months.

Precautionary statements

P234 Keep only in original container.

P260 Do not breathe dust/fume/gas/mist/vapours/spray.

P262 Do not get in eyes, on skin, or on clothing.

P281 Use personal protective equipment as required.

P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes.

Remove contact lenses, if present and easy to do. Continue rinsing.

P337+P313 If eye irritation persists: Get medical advice/attention.

P401 Store in accordance with local/regional/national/international regulations.

P501 Dispose of contents/container in accordance with local/ regional/ national/ international regulations.

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