DAYDROP

LUBRICANT AND HUMECTANT EYE DROPS

DESCRIPTION

DAYDROP is a humectant and lubricant sterile ophthalmic solution based on ectoine, carboxymethylcellulose and sodium hyaluronate in the form of a hydrogel. DayDrop is a sterile eye drop, pH-buffered, free of phosphates and without preservatives.

COMPOSITION

DAYDROP is formulated with ectoine, carboxymethylcellulose, sodium hyaluronate, pharmacopeia quality salts and purified water:

MODEL	DAYDROP
Content	%(w/w)
Ectoine	2
Sodium hyaluronate	0.1
Carboxymethylcellulose	0.15
Disodium tetraborate decahydrated	0.1
Boric acid	0.8
Purified water	q.s. 100

PACKAGING

DAYDROP is supplied in single-use containers made of low density polyethylene containing 0.4 mL of the product. Each box contains 30 single-use containers.

INTENDED USE

DAYDROP is an ophthalmic solution indicated to alleviate the ocular dryness and irritation produced by environmental factors (wind, dust, air conditioning, dry heat), prolonged use of screens or contact lenses and/or some pathologies that present the above mentioned symptoms.

PROPERTIES

DAYDROP alleviates eye dryness and dry eye perception.

DAYDROP alleviates eye irritation.

DAYDROP protects against external allergens alleviating allergy symptoms.

DAYDROP is compatible with contact lenses.

DOSAGE AND ADMINISTRATION

DAYDROP is administered by the ocular route (is applied on the eye surface).

Before opening a new DAYDROP container, ensure that the sealing of the plug is intact by checking the integrity of the safety

ring and the plug.

The solution must be used immediately after opening. In order to avoid tip or product contamination, do not touch the tip of the vial on the eye or other surfaces.

In case of people with difficult handling, or elderly people, it is recommended to receive support for the correct administration of the product.

The recommended dose is one or two drops in each eye, as many times is needed or according to the instructions received by the specialist.

DIRECTIONS FOR USE

1. Wash and dry hands before using.
2. Remove a single-use vial from the strip.
3. Verify that the vial is not chipped or cracked.
4. Turn over the plug to open the vial.
5. Remove the plug.
6. Tilt your head back and look up.
7. Gently pull the lower eyelid to form a small bag between the eyelid and the eye.
8. Hold the container with the other hand between the thumb and the middle finger and gently press to drop one or two drops of DayDrop into the eyelid bag.
9. Blink several times to allow drop distribution all over the eye.
10. If the drop has not entered, apply again.
11. Carefully wipe any rest of the product that may be around the eyelid with gauze.
12. Repeat the procedure with the other eye.
13. Discard the single-use vial (once opened do not re-use the vial for the next doses).

CONTRAINDICATIONS

Do not administer DAYDROP if you have hypersensitivity to hyaluronic acid or other components of the product.

WARNINGS AND PRECAUTIONS

- Do not use if you are allergic to any ingredient.
- Do not touch the tip of the vial with the eye or any surface.
- Do not share the product with other people.
- It is possible to have blurred vision immediately after using the drops. Wait for clarity of vision before carrying out risky activities due to blurred vision.
- Do not use the product after the expiration date indicated on the box or on the packaging.
- Keep out of the reach and sight of children.
- Use with caution in pediatric population (product not evaluated

in children)

- For external use only, do not ingest.
- Consult your doctor, pharmacist or optician before using the product during gestation or breast-feeding.

INTERACTIONS

Do not use with other products containing quaternary ammonium salts such as benzalkonium chloride because sodium hyaluronate may precipitate in its presence.

After using DAYDROP, wait at least 15 minutes to apply any other product or medicine to the eye.

DAYDROP is compatible with contact lenses. After putting on the contact lenses and before applying the drops, it is recommended to wait about 30 minutes to avoid incompatibility reactions between the products used for the maintenance of contact lenses and DAYDROP.

ADVERSE EVENTS

DAYDROP is well tolerated.

In some cases it may cause temporary eye irritation, blurred vision, foreign body sensation or eye hyperemia. Tell your doctor or pharmacist if you notice any side effects related to using DAYDROP. If these effects persist or worsen, consult your doctor or pharmacist.

STORAGE

Store between 5°C and 25°C. Do not freeze. Keep the vials protected from light inside the box.

SELL BY DATE

Do not use after the date marked on the package.

MANUFACTURED BY & LEGAL MANUFACTURER

  **i+Med S.Coop.**
c/Albert Einstein 15, Pab. 15,
01510 Vitoria-Gasteiz
Álava (Spain)
www.imasmed.com

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