



NINGBO GREETMED MEDICAL INSTRUMENTS CO., LTD.

Add: 16F-1, Building 1, No.98 Chuangyuan Road, Hi-Tech Zone, 315042 Ningbo, Zhejiang Province, PEOPLE'S REPUBLIC OF CHINA

Tel:0086-574-87739070, 87722370, 87739297, 87711530

Fax: 0086-574-87722360, 87739075

Instruction for use of Ultraviolet Germicidal Lamp

Usage scope: school, hospital, kid garden, hotel, house, pet shop, factory, restaurant ect.

| Powder | Length | Usage palce | Honder length | Ultravoilet Intensity |
|---------------|---------------|--------------------|----------------------|------------------------------|
| 6W | 212mm | <15 m ³ | 232mm | 18uv/m ³ |
| 8W | 288mm | | 314mm | 28uv/m ³ |
| 10W | 331mm | | 350mm | 30uv/m ³ |
| 15W | 436mm | 15 m ³ | 460mm | 44uv/m ³ |
| 20W | 589mm | 20 m ³ | 615mm | 75uv/m ³ |
| 30W | 894mm | 30 m ³ | 915mm | 107uv/m ³ |
| 40W | 1199mm | 40 m ³ | 1220mm | 117uv/m ³ |

Caution:

- 1. No people and pet on the space when on work of the Ultraviolet Germicidal Lamp.**
- 2. At least 30 mins of the sterilization time.**
- 3. Open door and window to fresh the air.**
- 4. Clean the lamp tube before use.**
- 5. Donot directly eye contact of the lamp.**
- 6. Use ozone free of ultraviolet germicidal lamp if have kids, old people and pregnant women.**