

# Instruction for use

**[Product name]:** Disposable Suction Catheter

**[Intended use] :** For Suction Catheter of phlegm when the patient cannot cough out the phlegm.

**[Specification] :** F6, F8, F10, F12, F14, F16, F18, F20. Fr22

Specifications	F6	F8	F10	F12	F14	F16	F18	F20
Outer dimension(mm)	2	2.7	3.3	4	4.7	5.3	6	6.7
Color of foveation tie-in	Pale green	Light blue	Black	White	Green	Orange	Red	Yellow

**[Instruction use]:**

Chose the suitable catheter, fold the back end of tube to avoid crushing patient' s mucous membranes and cause injury, then use sterile haemostatic forceps to insert the distal end to larynx ,then release the back end of tube, and suck up phlegm, when finished, turn off the negative pressure suction machine, take out the tube and discard the tube immediately.

**[ Caution]:**

1. For single use only,discard after use.Re-sterilization is strictly prohibited.
2. Do not use if package is open or damaged.
3. If Product re-use, it may be result in re-infection or cross-infection.
4. The product contains DEHP, it is not suitable for children, pregnant or nursing woman to use.
5. The storage environment shall be kept dry and kept away from sunlight.
6. Shelf life is 3 years. Do not use after 3 years.

**[Storage and transportation]:**

1. The storage environment shall be kept dry and kept away from sunlight.
2. The device shall be taken care during transportation and handling.