

## **Directions for use**

**-Application:**

By using oxygen or oil free compressed air as the power source, to use as treatment for Atomization inhalation

**-Use directions :**

Check the products to make sure it is well sealed. And also check if it is in term of validity. If everything is OK, tear the package open, and connect each part well. And check the connection firmness and air tightness; connect the oxygen mask with other power supply and get the power on and use it. Others instructions should be implemented with guidance of doctors.

**-Contraindication:**

None

**-Cautions:**

- 1) Only for single use, and after use, they should be discarded by medical units.
- 2) Do not use the products if the package has been broken.
- 3) Do not use the products if they are expired.
- 4) Do not use the products if patients are allergic to the raw material.
- 5) see the production date in the sealing of the packing and out packing box.

**-storage condition:**

The products should be stored indoors where it is clean, and well-ventilated, the related humidity is not more than 80%, and without corrosive gas.

**-life time:**

5 years from the date of EO sterilization.