

Instruction for use

Direction For Use:

1. Attach the oxygen supply tubing to the oxygen source.
2. Set the oxygen to the desired flow.
1. Check for flow through the device. Airways verify proper function fo the valves: The valve between the mask and the reservoir should rise on inspiration and lower on exhalation. The valve located on external mask surface should open during exhalation.

Warning: Always check oxygen flow through inlet valve before placing the mask on the patient.

4. Place the mask on the patient`s face with the elastic strap below the ears and around the neck. Gently pull the strap ends until the mask is secure. Mold the metal strip on the mask to fit the nose.

CAUTION:

1. For single patient use only.
2. Not intended for reprocessing.
1. Avoid freezing and excessive heat.
2. Be sure all connections are secure.