

## Instruction for use

1. Go to the local hospital. the maximum oxygen pressure should not exceed 16kpa (feel Microsoft, about 0.1g / cm<sup>2</sup>)
2. The oxygen absorption can be adjusted by plastic switch, and the pressure in the bag can be pressurized outside the bag is too low.
3. The oxygen bag is required to fill oxygen, which can be used continuously for about 1 hour when the instrument has low oxygen absorption.

### Caution:

- 1.The oxygenation pressure should not exceed 16kpa
2. When filling the oxygen needs to be stopped, please close the switch or close the pipeline, and store it in a cool, dry and ventilated room. The product should not be placed in the sun or at a high temperature, and do not contact with corrosive substances.