

HEMOTREAT H - Suppository



COD Presentation form No. pcs./individual box
HTS Suppository 12

COMPOSITION

Aloe oil extract, Calendula oil extract, Yarrow oil extract, Camphor, Ethanol, Clove essential oil, Semi-synthesis glycerides (Witepsol H15), Phenoxyethanol, Ethylhexylglycerin, BHA

| No | CAS | Ingredient | Function |
|----|------------|---|------------------------------|
| 1 | 85507-69-3 | Aloe oil extract | Protectant, Emollient |
| 2 | 84776-23-8 | Calendula oil extract | Protectant, Emollient |
| 3 | 8022-07-9 | Yarrow oil extract | Protectant, emollient |
| 4 | 76-22-2. | Camphor | Cooling, analgesic |
| 5 | 64-17-5 | Ethanol | Antiseptic, aromatic |
| 6 | 8000-34-8 | Clove essential oil | Analgesic, anti-inflammatory |
| 7 | 67701-26-2 | Semisynthesis glycerides (Witepsol H15) | Suppository base |
| 8 | 122-99-6 | Phenoxyethanol | Preservative |
| 9 | 70445-33-9 | Ethylhexylglycerin | Preservative |
| 10 | 121-00-6 | BHA | Antioxidant |

PROPERTIES

Lubricant, favoring hemorrhoids decreasing
Soothing the itching sensation
Reduces pain and local inflammation
Creates a protective film over the anal mucosa

INDICATIONS

HemoTreat H suppository is one of the most efficient way to improve, treat and prevent internal hemorrhoids and anal fissures.
Also, the unpleasant symptoms are quickly and efficiently controlled due to the properties of product ingredients



USAGE

HemoTreat H suppositories are administered internally, in the anus, after local cleaning, preferably after defecation.

The administration is made with clean and disinfected hand, applying light pressure, without forcing the entrance of suppository into the anal channel. For an easier application HemoTreat H suppositories are kept in the fridge. It is recommended to avoid defecation in the first 3 hours after the internal application of the suppository.

Use one suppository twice a day, in the morning and in the evening, for 5-7 days.

The time of use depends on the evolution of symptoms and age of disease. For subsequent prevention use one suppository a day, until total elimination of symptoms, but not for more than 30 days continuously.

USAGE CAUTIONS

HemoTreat H suppository is not used by people allergic to one of the components

If signs of local infections occur, the treatment will continue only with the doctor's advice

HemoTreat H suppository is not used for more than 30 days

Pregnant and nursing women should use HemoTreat H suppository only with the doctor's advice

Do not administer HemoTreat H suppository to children, only after seeing the doctor

Check the packaging integrity before usage

The product shall not be used if the packaging is damaged

Do not use after the expiry date written on the packaging

Keep away from children

RECOMMENDATIONS FOR THE PREVENTION OF THE ANO-RECTAL AFFECTIONS

Prevention of the anal-rectal diseases can be done by diet and an adequate lifestyle, which have as a result producing softer defecations:

Add as much fibers as possible to your diet (fruits, vegetables, grains, etc)

Drink an adequate quantity of water (8 -10 glasses/day)

Do not push the fecal bolus, but avoid prolonged defecation

Avoid intense wiping after defecation (to prevent local irritations)

Exercise at least 30 minutes daily

Regulatory status:

II a Class medical device according to Directive 93/42/EEC Last revision: august 2017

