

KLFMED[®]**Wenzhou K.L.F. Medical Plastics Co., Ltd.****CE Technical File****3-way Stopcock****Chapter 09. Instruction for Use**

Doc.No: KLF/TF-MDD-IX-09

Name:	Written By: Fengbo Lou	Reviewed By: Jianzhong Huang	Approved By: Yuewen Jiang
Date:	1 st Jan. 2021	1 st Jan. 2021	1 st Jan. 2021

Instruction for use

THIS GUIDANCE IS NOT PART OF THE PRODUCT AND IS PRESENTED CUSTOMERS ON REQUEST

Read the instructions carefully.

1. Indications for Use

3-way stopcock are the stopcock with 3 ways including protective caps, the seat of stopcock, the element of stopcock and T knob, which made of PC, PE and PP. They are intended use to connection infusion sets or extension sets or other plastic medic devices for distribution of solution.

2. Precautions

Always check for leaks after changing the connection.












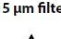


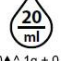


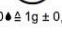
3. Cautions

Dispose of the 3-way stopcock safely, so that nobody can prick or injure themselves on it. Discard the products after each use. Do not clean or re-sterilize. This device remains sterile until the package is opened or damaged. Do not use if the sterile package has been opened or damaged. Never use if any part is disconnected or if protective cap is disconnected.

4. How to use

1. Open the individual package.
2. Connecting connectors to infusion sets or extension sets or other plastic medic devices and fix it.
3. Distribution of solution by T knob according to routine method.

5. Used symbol on blister

 REF	Product reference		For single use only		STERILE EO	Sterilized by Ethylene oxide
 LOT	Batch/Lot number		Do not use if unit package is damaged		PHT/DEHP	with content or presence phthalates
	Date of Manufacturing		Keep away from sunlight		15 µm filter	Filter of liquid with pore size
	Use by/Expiry date		Keep away from rain		15 µm filter	
	Manufactured by		Pyrogen free		20 drops/ml	The number of drops per ml
	See instruction for use		Temperature limitation		20 ± 1g ± 0.1g	

6. Re-sterilization

Product is not intended for re-sterilization. Manufacturer prohibits re-sterilization.

7. Reuse of product

Product is not intended for reuse. Product intended to be used once only for a single patient. Manufacturer prohibits reuse. Because when reuse is not guaranteed sterility

and purity of the product (implication – contamination, infection, injury of patient).

8. Manufacture and EC Authorized Representative

Manufacturer:

Name: Wenzhou K.L.F. Medical Plastics Co., Ltd

Add: No.29 Gangqiang Road, Airport New Area, Yongxing Street,
Longwan District, Wenzhou, Zhejiang, 325000, P.R.China

Mail: KLF@klfmed.com

Tel: 86-0577-86650005

Fax: 86-0577-86650007

EC Authorized Representative

Name: Shanghai International Holding Corp. GmbH (Europe)

Add: Eiffestrasse 80, 20537 Hamburg, Germany

Tel: 0049-40-2513175

Fax: 0049-40-255726