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Model: Hemo Treat H (Suppository)

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# HemoTreat™ H

HemoTreat H reduce pain and local inflammation, until their total disappearance, relieves pain and itching sensation, restores and maintains normal function of the anal mucosa

HemoTreat™ H

# HemoTreat™ H

Suppository for hemorrhoids and anal fissures

12 suppositories

Medical device for rectal use, class IIa, according with the Directive 93/42/CEE

**VALIDITY:**

Do not use after expiration date  
Store at 4-25°C

LOT: 3HTS048  
EXP: 2021/04

Medical device for rectal use, class IIa, according with the Directive 93/42/CEE

**PRODUCER:**  
S.C. GlobalTreat SRL,  
str. Fortuță, nr. 157, Galați, Romania  
Tel.: 0236 460361; 0372 350218  
Fax: 0236 460766  
www.hemotreat.eu  
contact@globaltreat.ro  
1370  
CE

REF HTS  
6 426282 940089

by people allergic to one of the components • If signs of local infections occur the treatment will continue only with the doctor's advice • HemoTreat H suppository is not administered internally, in the anal, HemoTreat H suppositories are eliminated of symptoms, but not for subsequent prevention use one suppository a day, until total symptoms and age of disease • For

**USAGE:**  
HemoTreat H suppository after administration internally, in the anal, after local cleaning, preferably after defecation • The administration is made with clean and disinfected hand with the doctor's advice • Do not administer HemoTreat H suppository to children, only after seeing the doctor • Check the packaging integrity before usage • The product shall not be used if the packaging is damaged • Do not use after the

package insert for full prescribng

the morning and in the evening

Use one suppository twice a day in the morning and in the evening

suppositories are left in the fridge • The product HemoTreat H application into the anal channel • For an easier insertion of suppository into the anal channel, without applying light pressure, without forcing the entrance of suppository into the anal channel, Calendula oil extract, Yarrow oil extract, Camphor Ethanol, Clove essential oil, Semi-synthetic glycerides (Wiposol H15), Phenoxylethanol, Erythroxylperin, BHA

**INGREDIENTS:**  
Aloe oil extract, Calendula oil extract, Yarrow oil extract, Camphor Ethanol, Clove essential oil, Semi-synthetic glycerides (Wiposol H15), Phenoxylethanol, Erythroxylperin, BHA