



Manifold set Instruction for use

Warning

Instruments are sterilized with ethylene oxide.

The tool must not be reused. Reuse may alter the quality of the instrument or cause cross-contamination

Description:

+ Is a plastic version with 2 or 3 screws. The two plastic ends are two connectors, one male and one female.

+ Each screw is attached with 1 female connector.

+ For the 2-port manifold, the female wrench is connected to the pressure monitoring wire and the contrast transmission line.

+ For the 3-port manifold, the female screw is connected to the pressure monitoring wire, the infusion line and the contrast line.

Point:

Used as an intravenous line for contrast media, drugs, and monitoring of arterial pressure during the procedure.

Instructions for use:

+ Open the package. Before opening the packaging, it should be observed that the packaging is not torn or damaged. If so, the tool must not be used

+ The female end of the manifold is attached to the distal end of the catheter, the male end is attached to a syringe



Kimal Plc

Arundel Road, Uxbridge , Middlesex, UB8 2SA England

Tel: +44 (0) 845 4379540 Fax: +44 (0) 845 4379541

www.kimal.co.uk

DCR 904



0088