



ZIBO EASTMED HEALTHCARE PRODUCTS CO., LTD
CE TECHNICAL FILE

Sterile Infusion Sets for Single Use



April, 2016

CE Technical File- Sterile infusion sets for single use	File No	DM/QJ-CE-016
	Revision	Rev. A/0
	Page	2 of 2

1 Instruction for Use

User Manual

[Product Name] Sterile Infusion Set for Single Use

Sterile Infusion Set for Single Use (hereinafter referred to as infusion set) is intended for clinical intravenous infusion solution.

[Specification]

IS-V, IS-NV.

[Structure and Character]

1. Infusion set consists of puncture closure devices, protective cover, air filter, hoses, dropper, drip bucket, flow regulator, liquid filter, comprising intravenous infusion needle.
2. Syringe sterilized by EO, should be sterile, non-toxic, pyrogen-free, non-hemolytic reaction.

[Methods]

1. Immediately use after unsealed, First remove the two sets of side protection, puncture device will be inserted into infusion bottles (bags), the schedule despite the gas and then venipuncture.
2. Use flow regulator, fluid flow in accordance with requirements.
3. Before use, intravenous infusion needle to the needle seat cover filter liquid tight joints to prevent leakage.



[Notice]

1. For single use only, discard after use.
2. Use before checking, do not use if the packaging is damaged, defiled.
3. The operation should be under professionals..

[Shelf life] 5 years

[Storage] Store in the relative humidity of no more than 80%, as well as non-corrosive and well-ventilated room.




Zibo Eastmed Healthcare Products Co., Ltd.

No.118, Huaguang Road, Zhangdian District, Zibo City, Shandong Province, China



STERILE EO

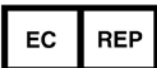

SUNGO EUROPE B.V.

Olympisch Stadion 24, 1076 DE Amsterdam, Holland.

Date: Apr. 15, 2016

Version: A/0