NOTICE TO USER

If a serious incident occurs in relation to this device, the incident should be reported to the manufacturer and to the appropriate competent authority of the member state in which the user and/or the patient is established. To report to the manufacturer, see the contact information provided in the Customer service section or Technical assistance section of these instructions.

For laboratory professional use only.

INTENDED USE

For In Vitro Diagnostic Use. The Abbott RealTime High Risk HPV Controls are used to establish run validity of the Abbott RealTime High Risk HPV assay when used for the detection of high risk human papillomavirus (HPV) DNA in clinical specimens.

INTENDED USER

The intended users for the Abbott RealTime High Risk HPV assay are laboratory and healthcare professionals.

CONTENTS

- CONTROL** – Abbott RealTime High Risk HPV Negative Control (List No. 2N09Z) (12 vials, 0,5 mL per vial). <0.01% noninfectious DNA with Beta Globin sequence in a buffered solution with carrier DNA. Preservatives: sodium azide and 0.15% ProClin 950.
- CONTROL** + Abbott RealTime High Risk HPV Positive Control (List No. 2N09A) (12 vials, 0,5 mL per vial). <0.01% noninfectious DNA with HPV and Beta Globin sequences in a buffered solution with carrier DNA. Preservatives: sodium azide and 0.15% ProClin 950.
- The Abbott RealTime High Risk HPV Control Kit must only be used with the Abbott RealTime High Risk HPV assay (List No. 2N09).

STANDARDIZATION

Standardization is achieved by manufacturing to a pre-determined concentration of each plasmid.

For these linearized plasmid targets, the qualified starting materials are purified plasmids that are received from a certified vendor or vendors:

- HPV Positive Control plasmids:
 - HPV 16 DNA Plasmid
 - HPV 18 DNA Plasmid
 - HPV 58 DNA Plasmid
 - Beta Globin DNA Plasmid
- HPV Negative Control plasmids:
 - HPV Internal Control DNA Plasmid

PRECAUTIONS

- IVD**
- Do not use beyond expiration date.

Components of the Abbott RealTime High Risk HPV Control Kit (List No. 2N09-80) contain the following components:

- 2-Methyl-2H-isothiazol-3-one
- Sodium azide

The following warnings apply:

	Warning	
EUH032	Contact with acids liberates very toxic gas.	
H317	May cause an allergic skin reaction.	
P261	Avoid breathing mist / vapours / spray.	
P280	Wear protective gloves / protective clothing / eye protection.	
P272	Contaminated work clothing should not be allowed out of the workplace.	
P302+P352	IF ON SKIN: Wash with plenty of water.	
P333+P313	If skin irritation or rash occurs: Get medical advice / attention.	
P362+P364	Take off contaminated clothing and wash it before reuse.	
P501	Dispose of contents / container in accordance with local regulations.	

−10°C

SHIPPING CONDITIONS

Ship on dry ice.

TECHNICAL ASSISTANCE

For technical assistance, call Abbott Technical Services at 1-800-553-7042 (within the US) or +49-6122-580 (outside the US), or visit the Abbott website at www.molecular.abbott.

	Abbott GmbH Max-Planck-Ring 2 65205 Wiesbaden, Germany
--	--

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© 2008, 2021 Abbott. All Rights Reserved.