



Advanced Nasal Spray
Help To Protect From Virus

Nitrovid

Advanced Nasal Spray
Help To Protect From Virus

Nitric Oxide
Releasing Solution
Kills 99.9% Virus

R.V GROUP **CE** **30 ml**

Each ml contains:
Each ml of Nitric oxide releasing solution of Sodium Nitrite.

Composition:

Device A: Sodium Benzoate USP	0.5% w/v
Device B: Citric Acid USP Sodium Chloride USP Benzalkonium Chloride USP	0.02% w/v 0.2% w/v 0.02% w/v

Excipients: q.s

Directions:
Hold upright and squeeze in each nostril.

Dosage:
2-3 sprays per nostril for protection from virus within 6-8 hours.

Indications For Use:
• 2-3 times per day for those who have high risk of getting infected or in contact with those who have been infected.
• Or as directed by physician.

Storage:
Store in a dry place at a temperature not above 30°C (86°F) in light.
Do not use if seal is broken.
Keep out of reach of children.
For more information refer package insert.

Directions For Use:
Blow your nose gently. Wash and clean your hands before and after use.
1. Hold the bottle upright and spray 2-3 times on blue part of both sides of the nozzle and press at the same time.
Before first time use or if not used for a long time, spray 2-3 times to activate the solution.
In case the spray nozzle is blocked clean it under hot running water and dry it properly with clean tissue. Insert the nozzle into the nostril and please vertically upward and spray please allow the excess solution to trickle down.
Repeat the same for other nostril. Do not spray continuously for more than 10 seconds under hot running water, dry it properly with clean tissue and put the plastic cap back or do not use the product cap back if it contains any liquid or disinfectant.

Step 1: Blow your nose gently.
Step 2: Wash and clean your hands.
Step 3: Hold the bottle with your forefinger and middle finger on either side of the nasal applicator and your thumb underneath the nozzle.
Step 4: Close 1 nostril, tilt your head forward slightly and, keeping the bottle vertically upward carefully insert the nasal applicator into the nostril. Press the nozzle and press firmly and quickly press 1 time on the applicator to release the spray.
Step 5: Repeat steps 3 and 4 for the other nostril.
Step 6: Breathe out through your mouth.
Step 7: Repeat steps 3 through 5 in the same order for the second nostril.
Step 8: Wipe the nasal applicator with a clean tissue and replace the dust cap.

Nasal Spray

MNI
Omnicells Pharma (Pvt.) Ltd.

Nitrovid

Advanced Nasal Spray
Help To Protect From Virus

Nitric Oxide
Releasing Solution
Kills 99.9% Virus

R.V GROUP **CE** **30 ml**

Technology developed by:
Finaso Pharma Pvt. Ltd.
H. No. 8-B, Building No. 450,
Phase No. 1 & 2, Opp. Bharat Biotech,
MS Sectors 9 & 10, Gurgaon, Haryana,
Phase - I, Tukapally, Shramipet (M),
Medchal-Malkajgiri Dist,
Telangana State, India.

Manufactured at:

VIVIMED LABS LTD.
Plot No. 8, Phase V, DA, Jeedimetla,
Jeedimetla (V),Dhulballepur (M),
Medchal - Malkajgiri (Dist), 500055.

Manufactured for:

MNI
Omnicells Pharma (Pvt.) Ltd.
Aneraj (W), Mumbai-400053, India.

Manufacturing country: India

Mfg. Hs. No.: 168/10/169/CC
Batch No.:
Mfg. Date:

Barcode

FOR EXTERNAL USE ONLY

Nitric Oxide

PATENTED DUAL CHAMBER TECHNOLOGY

Nitrovid
Advanced Nasal Spray
Help To Protect From Virus

Nitrovid
Advanced Nasal Spray
Help To Protect From Virus

Dimensions: 65 mm x 40 mm x 135 mm



Front

Nasal Spray



Omnicals Pharma (Pvt.) Ltd.

Nitrovid

Advanced Nasal Spray
Help To Protect From Virus

Nitric Oxide Releasing Solution

Kills 99.9% Virus




30 ml



Back

Each ml contains:
Each ml of Nitric oxide releasing solution contains 5 mg of Sodium Nitrite.

Composition:

Device A: Sodium Nitrite USP 0.5% w/v
Benzalkonium Chloride USP 0.02 % w/v

Device B: Citric Acid USP 0.2% w/v
Sodium Chloride USP 0.9 % w/v
Benzalkonium Chloride USP 0.02% w/v
Expipients q.s.

Directions:
Hold upright and squeeze in each nostril.

Dosage:

- 1-2 sprays per nostril for protection from virus within 6-8 hours
- 2-3 times per day for those who have high-risk to get the virus transmission or in case already exposed to virus infection
- Or as directed by physician

Storage:
Store in a dry place at a temperature not exceeding 30°C.
Protect from light. Do not use if seal is broken.
Keep out of reach of children.
For more information refer package insert.

Technology developed by:
Finiso Pharma Pvt. Ltd.
H. No. 6-8, Building No. 450, Room No.: 101 & 301,
Opp. Bharat Biotech, MS Science & Technology Park,
Phase - I, Turkapally, Shamirpet (M),
Medchal-Malkajgiri Dist, Telangana State, India.

Manufactured at:

 **VIVIMED LABS LTD.**
Plot No 8, Phase V, DA, Jeedimetla, Jeedimetla (V), Outhbullapur
(M), Medchal – Malkajgiri (Dist), 500055.

Manufactured for:

 **Omnicals Pharma (Pvt.) Ltd.**
Andheri (W), Mumbai-400053, India.

Manufacturing country: India

Mfg. Lic. No.: 168/HB/AP/95/F/GC

Batch No.:

Mfg. Date:

Exp. Date:

FOR EXTERNAL USE ONLY




Nitric Oxide

**PATENTED
DUAL CHAMBER
TECHNOLOGY**



Barcode

Dimensions: 35 mm x 55 mm