

## INSTRUCTION FOR USE

### POSE-CREZOL

*Pose Health Care Limited/ Thailand*

**Address: 1 Soi Ramintra 107, Ramintra Road, Kannayao, Bangkok  
10230, Thailand**

#### **Ingredients :**

+ Active ingredients:

- Phenolic compound 45.0% w/w

+ Other ingredients:

- Sodium Lauryl Sulfate 15.0% w/w

- Sodium hexametaphosphate 20.0% w/w

- Talcum 20.0% w/w

**Indication:** Moderate level of disinfection for Medical instrument.

#### **Instruction for use:**

- Step 1: Mix 1 sachet (5 grams) with 1 litre of clean water (Tap water) or Mix 1 sachet (25 grams) with 5 litres of clean water (Tap water).
- Step 2:
  - Cleaning Medical Equipment: Feces, mucous, tissues, blood and other body fluids must be thoroughly cleaned from surfaces and lumens of equipment by Enzymatic detergent (TERGEZYME-PLUS) before processing in POSE-CREZOL.
  - Thoroughly clean, rinse and rough dry equipment before immersing in POSE-CREZOL. Clean and rinse the lumens of hollow equipment before filling with POSE-CREZOL
- Step 3: Disinfection

+ Method 1:

- Immerse cleaned and rough-dried medical equipment completely in the POSE- CREZOL.
- Leave medical equipment completely immersed for 10 minutes.
- At the end of the required time remove medical equipment from the solution using the aseptic technique.
- Rinse thoroughly with the appropriate quality of water (sterile or potable) following the rinsing instructions.
- Thoroughly rinse the medical equipment by immersing it completely in three separate copious volumes of water. Each rinse should be a minimum of 1 minute in duration. Use fresh portions of water for each rinse. Discard the water following each rinse.

+ Method 2:

- Wiping or spray POSE-CREZOL solution on the surface and let it dry (for medical devices which need to disinfect surface only)

*Note: Mixed solution can be used within 1 day*

**PERSONAL PROTECTION**

Wear chemical resistant gloves

Wear face mask covering mouth and nose

Wear gloves to protect nose

**Contraindication:**

None

## **Possible adverse effects**

**EYES:** Hold eye open and flush slowly and gently with water at least 15 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue flushing. Call a poison control center or doctor for treatment advice.

**SKIN:** Take off contaminated clothing. Wash skin with soap and plenty of water, call a poison control center or doctor for treatment advice.

**INHALATION:** Remove to the fresh air. If breathing is difficult, get medical attention.

**INGESTION: DO NOT INDUCE VOMITING.** If swallowing much amount should to get medical attention.

## **Storage:**

Storage at room temperature, in original packing, far away from high temperature or flame

## **Packing:**

5g and 25g sachet

## **Shelf life:**

3 years from manufacturing day



*Chotcomwongse*