

**DIRUI**<sup>®</sup>

# Probe Cleanser

## Instruction

**【Product Name】** Probe Cleanser

**【Packaging Specifications】** 50mL, 4x50mL

**【Intended Use】** It is used for cleaning the detection part of instrument, rotary valve, whole blood suction pipe, hemoglobin cuvette, prevention of protein build-up.

**【Test Principle】** As a strong alkaline cleaner for the removal of cellular liquid and proteins remaining in the Automatic Hematology Analyzer.

**【Main Composition】** Sodium hypochlorite.

**【Storage Conditions and Shelf Life】**

When stored at 2°C~30°C and protected from light, and the shelf life refers to the label.

**【For Instrument】** BCC series Automatic Hematology Analyzer.

**【Method】** Refer to the related Automatic Hematology Analyzer operating instructions.

**【Performance Index】** pH is 12.30±0.30 at (25±1)°C












**【Matters Needing Attention】**

1. Please use within 15°C~30°C.
2. Avoid contact with the skin and eyes. Once in contact with the skin, flush the area with water. Once in the eye, wash immediately with plenty of water and receive treatment. If you accidentally take it, please see your doctor promptly.
3. The product contains sodium hypochlorite, and the hazardous label is marked on the product packaging in accordance with the requirements of EU Parliament and Committee regulations.  
R31: Contact with acid will release toxic gases  
R36/38: Irritating to eyes and skin  
S50: Do not mix with acidic solution  
S24/25: Preventing contact with eyes
4. Samples, waste liquids, etc. are potentially biologically infective. Operators should comply with laboratory safety operation regulations and dispose of waste liquids in accordance with local medical waste, infectious waste, industrial waste, etc.

**【References】**

Li Jingfen. Effect of additives on the stability of sodium hypochlorite and peracetic acid [J]. Heilongjiang Medical Journal, 2001(2):8

**【Explanation for Symbols】**

 Please read instruction	 Storage temperature	 Keep in Dark Places	 Manufacturer	
 Lot No	 Expiry	 Catalogue number	 Danger level	 In vitro Diagnostic Reagent
 European In Vitro Diagnostic Medical Device Directive 98/79/EC(IVDD)			 Authorised Representative	

**【Date of Approval and Revision】** 02/2019



**DIRUI INDUSTRIAL CO., LTD.**

95 Yunhe Street, New & High Tech.

Development Zone Changchun, Jilin 130012 P.R.China

Tel: +86 431 85100409

Fax: +86 431 85172581

E-mail: dirui@dirui.com.cn

http://www.dirui.com.cn



**EC REP**

**EMERGO EUROPE** Prinsessegracht 20  
2514 AP The Hague The Netherlands