

Ningbo Greatcare Trading Co., Ltd.

Unit 93, Building 12, No. 818, Qiming Road,
Yinzhou, 315105 Ningbo, Zhejiang, China.

Tel: +86-574-8778 6632
Fax: +86-574-8778 6644
info@greatcare.cn



Creating a Better Care

EN

NASAL CANNULAS FOR OXYGEN THERAPY INSTRUCTIONS FOR USE

INDICATIONS:

Therapeutic administration of oxygen through the nose.
NB: Disposable product – **do not reuse**.
The oxygen source must be in according to the regulations in force.
Device must be connected and actived by qualified personnel.

INSTRUCTIONS

Introduce the nasal distributor carefully in each one of the nostrils and proceed as described underneath for each model of nasal cannula.
Keep the nasal distributor in position and pass both the right and left tubing over the ear-bows down over the jaws to under the chin. Adjust the tubing by means of the plastic ring.
Plug the end of the connection tubing onto the oxygen source. When the device is properly dressed, deliver oxygen and regulate the flow as necessary.

CONTRA-INDICATIONS:

No contra-indications are described.

WARNINGS:

If device is reused, contrarily to the present instructions, this may compromise following:

1. alteration of materials
2. Not cleanliness of the device and possible presence of biological residues that might cause cross-infections
Loss of initial functional features of product. Before connecting to oxygen source verify its functioning. The accidental crushing of the connection tubing do not allow oxygen to flow properly and may cause the disjunction of the connector. It is therefore recommended that the users control carefully. The same attention must be paid even to the models which are supplied with a sure-flow tubing for which the risk of disjunction exists even though in a reduced way.

STORAGE

The package must be stored at a temperature between 0°C and 50°C and relative humidity between 20% and 80%.

GUARANTEE - NOTICE

FIAB guarantees that the product complies with Directive 93/42/EEC and has been manufactured according to the procedures of FIAB Quality System certified ISO 13485.

No responsibility may be ascribed to the producer who shall not be held liable for medical costs, direct damage due to lacking function or malfunction of the above products, when used differently from the instructions for use. We recommend to report opportunely any malfunction or defect of the product to FIAB Quality Assurance Service.

WASTE DISPOSABLE

Waste coming from hospitals must be disposed in accordance with regulations in force.



GIÁM ĐỐC

Cao Xuân Quỳnh Anh

2013 / 01



0086

Ningbo Greatcare
Unit 93, Building 12, No.818, Qiming Road, Yinzhou, 315105 Ningbo, Zhejiang, China


Ningbo Greatcare Trading Co., Ltd.

Unit 93, Building 12, No. 818, Qiming Road,
Yinzhou, 315105 Ningbo, Zhejiang, China.

Tel: +86-574-8778 6632

Fax: +86-574-8778 6644

info@greatcare.cn

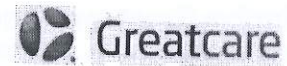

Creating a Better Care

IFU of oxygen cannula

Instructions for use

a Series of Oxygen Cannula are for single use only. After taken out from package, it shall be used immediately, and shall be destroyed after use;

b Do not use if the package broken;

c The storage shall avoid high temperature, high humidity, without erosive gas and with good aeration;

d The products are sterilized with EO, each items shall be inspected for conformity to standard before delivery;

e The shelf life of products are 5 years since sterilization.



Cao Xuân Quỳnh Anh