


↑  
OPEN

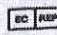
# TOPCARE

## NON-REBREATH MASK

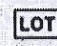
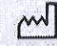

**DIRECTION:**

1. Place mask over patient's face, centering both nose and mouth.  
The elastic strap should be below the ears and around the neck.
2. Gently pull the ends of the strap to adjust for comfort and fit.
3. Adjust metal strip at bridge of nose to secure fit.
4. Attach the other end of O2 tube to oxygen source.

 Ningbo Shengyurui Medical Appliances Co., Ltd.  
No. 138 Binhaisi Road, Hangzhou Bay New Zone  
315336, Ningbo, Zhejiang Province, P.R. China

 Shanghai International Holding Corp.  
GmbH(Europe)Eiffestrasse 80, 20537  
Hamburg, Germany

Size: Pediatric Elongated

 19615  
 2019-12  
 2024-11

CE 0120  **STERILE EO** **MADE IN CHINA**



**GIÁM ĐỐC**

*Cao Xuân Quỳnh Anh*