

Ningbo Greatcare Trading Co., Ltd.

Unit 93, Building 12, No. 818, Qiming Road,
Yinzhou, 315105 Ningbo, Zhejiang, China.

Tel: +86-574-8778 6632
Fax: +86-574-8778 6644
info@greatcare.cn



Creating a Better Care

IFU of Feeding tube

DESCRIPTION

A feeding tube is a sterile, single-use tube manufactured in various sizes from medical grade PVC. It has four lateral eyes located at the end of the tube. The tube is used by directly inserted into the into stomach through nasal cavity and esophagus

INTENDED USE/ INDICATIONS

It is a medical device used to provide nutrition to patients who cannot obtain nutrition by mouth, are unable to swallow safely, or need nutritional supplementation

CONTRAINDICATIONS

- There are no known contraindications.

WARNINGS/PRECAUTIONS

- should be used by qualified medical personal only
- for single use only
- do not use if sterile package is opened or damaged

- Do not use after the expiry date.

DIRECTIONS FOR USE

- Peel off the package, and take out the tube
- Place suction tube at the root of the tongue and push it following gulps
- make sure that the tube is in stomach(by auscultation of upper abdomen after 20ml air inflation though the tube)
- fix the proximal end of the tube with adhesive tape or special fixator
- Control tube to syringe
- after feeding wash the feeding tube with 30-50ml of water and close connector's cap
- after manipulation extract the tube using disinfective swab

CAUTION

- Single use only.
- Sterile if package is unopened and undamaged.
- Store in a cool dry place.



GIAM ĐỐC

Cao Xuân Quỳnh Anh