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□ PROPERTIES:

HUMER DECONGESTIONNANT RHUME is based on a unique formula (patent pending) CAPTOMUCIL®⁽¹⁾, constituted by a natural chitosan polysaccharide.

HUMER DECONGESTIONNANT RHUME is useful in the treatment of respiratory affections thanks to 3 actions:

- Promotes elimination of excess secretions: when mixed with nasal mucin in certain proportions, CAPTOMUCIL®⁽¹⁾ is able to break the viscosity of mucus network
- Hypertonic solution reinforces CAPTOMUCIL®⁽¹⁾ decongestant effect: increase fluidity of excess mucus
- Soothes irritated nasal mucosae

□ INDICATIONS:

HUMER DECONGESTIONNANT RHUME is useful to treat respiratory affections in which mucus hypersecretion is a clinical complication, such as common cold and rhinitis that may involve in acute or chronic rhinosinusitis and allergic rhinitis.

□ INSTRUCTIONS FOR USE:

Adults and children > 12 years: 1-2 sprays in each nostril 3-4 times a day.

Children 6-12 years: 1 – 2 sprays in each nostril, at need, up to 2 times a day.

In children < 12 years of age use the product only after doctor advice.

□ WARNINGS AND PRECAUTIONS:

Side Effects

At doses necessary to achieve a therapeutic effect, the occurrence of side effects is not expected, except for cases of individual intolerance. If after 5-7 days of use the symptoms persist consult your doctor.

Interactions with dietary supplements and drugs

For the components of HUMER DECONGESTIONNANT RHUME, there are no known interactions with dietary supplements or drugs. However, as general rule, it is advisable to take HUMER Decongestant at least 1 hour before or after the use of drugs for nasal administration. Consult your physician before taking the product in case of concomitant drug therapies.

Warnings and precautions for use:

Do not use product in children under 6 years old.

In case of pregnancy or breast-feeding and in children < 12 years of age, use only after doctor advises.

Do not exceed recommended dosage.

Do not use the product continuatively for more than 30 days.

The product may have a reduced effectiveness in case of rhinosinusitis associated to nasal polyps.

Do not take in case of intolerance to the product.

Do not take in case of known hypersensitivity to the functional components or to some excipient.

In any case of unexpected adverse effects suspend the use and consult a doctor.

Depending on the nature of the product, a mild stinging sensation might be experienced during or for a short time afterwards the application.

After use, rinse with warm water and dry the nozzle with a tissue.

Expiring Date

Do not take after expiration date printed on the packaging

The expiry date refers to the product in its intact container when stored.

Do not use the product one month from the opening

Disposal

Expired device must be managed according to the waste management regulations of Local Entity Manager.

Do not dispose in any case the product or its packaging into environment.

□ STORAGE CONDITIONS:

Keep the product in a dry place and away from direct heat or light sources. Do not use in case of damaged package. Keep out of reach of children. Keep the device between 15 and 25°C.

□ TECHNICAL DATA:

NATURE	TECHNICAL CHARACTERISTICS
pH	5.50 – 5.90
Density	0.990 – 1.030 g/ml
Osmolarity	550 – 820 mOsM/Lt
Chitosan assay	≥ 0.187 mg chitosan/puff (0.13 ml) (≥90 %) Theoretical value : 0.208 mg chitosan/puff (0.13 ml)
TAMC	≤ 100 CFU / ml
TYMC	≤ 10 CFU / ml
Pseudomonas aeruginosa	Absence
Staphylococcus aureus	Absence
SHELF-LIFE	36 months

□ REGULATORY STATUS:

In European Union, "HUMER DECONGESTIONNANT RHUME" is a Medical Device, CE Marked (Class IIa).

Legal manufacturer: SIIT srl - via Ariosto 50/60 20090 Trezzano sul Naviglio (MI) - ITALY

Distributor: Laboratoires URGO HEALTHCARE – 42 rue de Longvic - 21300 CHENOVE - FRANCE

(1) CAPTOMUCIL® is SIIT's registered trade mark