COX

- I** - Ago coassiale (o introduttore) per biopsia ai tessuti molli.
- GB** - Co-Axial Introducer needle for biopsies on soft tissue.
- F** - Aiguille coaxiale pour biopsie des tissus mous.
- D** - Koaxialnadel (Einführungsnadel) für weiche Gewebe.
- E** - Aguja coaxial (o introductora) para biopsia de tejidos blandos.
- P** - Agulha de introdução coaxial para biópsias em tecidos moles.
- GR** - Ομοαξονική βελόνα εισαγωγής για βιοψίες σε μαλακό ιστό.
- NL** - Coaxiale naald (of introductienaald) voor biopsieën van zacht weefsel.
- FIN** - Koaksiaalinen biopsianeula (tai asetin) pehmytkudokseen.
- S** - Koaxial införingsnål för biopsier på mjuk vävnad.
- DK** - Co-Axial indføringsnål til blødvævnbiopsi.
- PL** - Współosiowy przewodnik igły do biopsji tkanek miękkich.
- CZ** - Koaxiální zaváděcí jehla pro biopsie měkkých tkání.
- SK** - Koaxiálna ihla aplikátora pre biopsie mäkkého tkaniva.
- SLO** - Koaksialna (ali uvajalna) igla za biopsijo mehkih tkiv.
- H** - Koaxiális introducer tű lágyrész biopsziákhoz.
- RU** - Коаксиальная игла для биопсии мягких тканей.

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GB

The Co-Axial mod. COX introducer needle is intended to be used when a multiple biopsy is required.

Used with a related istological biopsy needle with finer diameter, it allows multiple accesses to the biopsy site with a single puncture causing less trauma to the patient.

Designed to be used with MEDAX srl mod. UNIVERSAL, U. PLUS, U.DUO, FACILE, VELOX2, VELOX, CAESAR and BIOFEATHER biopsy devices (remember to remove the stopper on the devices, by sliding it of their cannula).

Refer to the table here below for the possible matching:

<i>CO-AXIAL NEEDLE (mod. COX)</i>	<i>RELATED BIOPSY NEEDLE</i>
13 G	14 G
15 G	16 G
17 G	18 G
19 G	20 G
20 G	21 G

It also serves as a general purpose aspiration biopsy needle for cytological examinations.

INSTRUCTION FOR USE:

- Slowly insert the needle into the tissue up to the biopsy area.
- Remove the cap and the stylet.
- In case of cytological examination, connect a syringe to the needle (luer lock cone) and aspirate the desired liquid quantity.
- When the COX is used as a co-axial needle, insert the related biopsy needle into the cannula and proceed with the sampling (refer to the instruction sheet of the related biopsy needle). Remove the related biopsy needle and insert it again to perform multiple biopsy.
- Slowly remove the Co-axial needle.
- At the end of procedure, remove all devices and accessories paying attention to the necessary precautions for potentially contaminated materials.
- Throw away after the use and discharge together with the hospital waste following, the laws in force.

LIMITATIONS AND WARNINGS:

- Usage allowed only to qualified physicians. MEDAX declines every responsibility for damages caused to the patient when the device is used in an improper way, with wrong manoeuvres or if handled by non qualified personnel.
- The procedure described is for guidance only: the use of the device must be based on the clinical training of each physician.
- non-pyrogenic, sterile device: not to be reused, sterilised or reconditioned in any way.
- Disposable single-use/single-patient: re-use of the disposable may cause cross infection to patient and/or user.
- Verify the integrity of the packaging and the validity of the product before use.
- The product must not be used if the package is unsealed, damaged or wet.
- Keep in a fresh and dry place, avoid the exposure to light and high temperature.

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