CHIBA-WESTCOTT-FRANSEEN-TURNER-QUINCKE

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INSTRUCTIONS FOR USE:

- Slowly insert the needle into the tissue up to the biopsy area.
- Remove the cap and the stylet.
- Connect a syringe to the needle (luer-lock cone) and aspirate the desired liquid quantity.
- Slowly remove the needle.
- At the end of procedure, remove all devices and accessories paying attention to the necessary precautions for potentially contaminated materials.
- Throw away after the use and discharge together with the hospital waste following, the laws in force.

NOTE : Westcott needle is provided with a side hole on the cannula. When the needle is inserted and the mandrel is removed, a rotating movement of the cannula allows the collection of a small quantity of tissue for istological analysis.

LIMITATIONS AND WARNINGS:

- Usage allowed only to qualified physicians. MEDAX declines every responsibility for damages caused to the patient when the device is used in an improper way, with wrong manoeuvres or if handled by non qualified personnel.
- The procedure described is for guidance only: the use of the device must be based on the clinical training of each physician.
- Non-pyrogenic, sterile device: not to be reused, sterilised or reconditioned in any way.
- Disposable single-use/single-patient: re-use of the disposable may cause cross infection to patient and/or user
- Verify the integrity of the packaging and the validity of the product before use.
- The product must not be used if the package is unsealed, damaged or wet.
- Keep in a fresh and dry place, avoid the exposure to light and high temperature.

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