HANDLEX

- I** - Ago per aspirato midollare.
- GB** - Bone marrow aspiration needle.
- F** - Aiguille de ponction médullaire.
- D** - Nadel für Knochenmarkansaugung.
- E** - Aguja para aspirado de médula.
- P** - Agulha de aspiração de medula óssea.
- GR** - Βελόνα αναρρόφησης μυελού των οστών.
- NL** - Opzuignaald voor beenmerg.
- FIN** - Neula luuytimen aspiraatioon.
- S** - Aspirationsnål för benmärgsbiopsi.
- DK** - Knoglemarvsaspirationsnål.
- PL** - Igła biopsyjna do szpiku kostnego.
- CZ** - Aspirační jehla na kostní dřeň.
- SK** - Ihla na nasávanie kostnej drene.
- SLO** - Igla za aspiracijo mozga.
- H** - Csontvelő aspirációs tű.
- RU** - Игла для аспирации костного мозга

8451610000/01 del 20/07/2012

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INSTRUCTIONS FOR USE:

- Adjust the projection of the cannula by turning the ring-nut.
- Slowly insert the needle in the tissue up to the bone marrow cavity (if possible, the needle is intended to be used in a inclined position).
- Remove the stylet.
- Connect a syringe to the needle and aspirate the desired bone marrow quantity.
- Remove the needle.
- At the end, remove all devices and accessories paying attention to the necessary precautions for potentially contaminated materials.
- Throw away after the use and discharge together with the hospital waste following, the laws in force.

LIMITATIONS AND WARNINGS:

- Usage allowed only to qualified physicians. MEDAX declines every responsibility for damages caused to the patient when the device is used in an unproper way, with wrong manoeuvres or if handled by non qualified personnel.
- The procedure described is for guidance only: the use of the device must be based on the clinical training of each physician.
- Don't move sideways the needle into the narrow without the mandrel into the cannula.
- Disposable, non-pyrogenic, sterile device: not to be reused, sterilised or reconditioned in any way.
- Disposable single-use/single-patient: re-use of the disposable may cause cross infection to patient and/or user.
- Verify the integrity of the packaging and the validity of the product before use.
- The product must not be used if the package is unsealed, damaged or wet.
- Keep in a fresh and dry place, avoid the exposure to light and high temperature.

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