SINUREP

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INSTRUCTIONS FOR USE:

- Fully insert the needle provided with the hook in the tissue and proceed to the depth of the area to landmark.
- Remove the needle to open and fix the hook. Keep the wire still until the landmark section disappears.
- At the end of procedure, remove all devices and accessories paying attention to the necessary precautions for potentially contaminated materials.
- Throw away after the use and discharge together with the hospital waste following, the laws in force.

WARNINGS AND LIMITATIONS OF USE:

- Usage allowed only to qualified physicians. MEDAX declines every responsibility for damages caused to the patient when the device is used in an unproper way, with wrong manoeuvres or if handled by non qualified personnel.
- The procedure described is for guidance only: the use of the device must be based on the clinical training of each physician.
- non-pyrogenic, sterile device: not to be reused, sterilised or reconditioned in any way.
- Disposable single-use/single-patient: re-use of the disposable may cause cross infection to patient and/or user
- Verify the integrity of the packaging and the validity of the product before use.
- Use immediately after removal from the primary packaging.
- The product must not be used if the package is unsealed, damaged or wet.
- Keep in a fresh and dry place, avoid the exposure to light and high temperature.

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