## EPITHEASY

- I** - Dispositivo a punzone per prelievo tessuto cutaneo
- GB** - Skin Biopsy Punch
- F** - Punch à biopsie cutanée
- D** - Hautstanze zur Entnahme einer Hautbiopsie
- E** - Punzón para biopsia cutánea
- P** - Punção de Biopsia Cutânea
- GR** - Συσκευή Βιοψίας Δέρματος
- NL** - Stans voor afname huidbiopt
- FIN** - Pistoneulalla varustettu laite ihokudosnäytteen ottamiseksi
- S** - Stans för hudbiopsi
- DK** - Hud biopsistanse
- PL** - Biopsyjne przyrządzenie sztancowe do pobierania tkanki skórnej
- CZ** - Jehla pro kožní biopsii
- SK** - Ihla na kožnú biopsiu
- SLO** - Pripomoček z iglo za odvzem kožnega tkiva
- H** - Lyukmetsző eszköz bőrszövet minta vételezésére
- RU** - Устройство для биопсии кожи

8451360000/02 del 20/07/2012

I

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GB

#### INSTRUCTIONS FOR USE:

- Before performing the skin biopsy, stretch the skin following its natural tension lines.
- Hold the device with your thumb and index fingers, then position it on skin surface. Press and turn clockwise, using moderate pressure. Depth of penetration is fundamental for obtaining a sufficient amount of sample. The physician should establish the correct penetration depth, taking into consideration the type of biopsy and tissue.
- Extract the device and gently lift the core using forceps, taking care not to tear the tissue.
- Cut the core at its base and put the sample into a suitable container.
- Some models are equipped with an ejector: after extracting the device activate the ejector in order to retrieve the sample and put it into a suitable container.

NOTE: the incision performed is oval and not circular due to the tension applied during biopsy: for this reason, significant haemostasis is unlikely.

Sutures are not necessary for incisions up to 4 mm, treatment with gauze and plasters is generally sufficient. With larger incisions, sutures or clips may be used, which in any case will not leave any evident scar.

#### WARNINGS AND LIMITATIONS OF USE:

- For use only by medical specialists. MEDAX shall not be held responsible for patient injury in case of improper or incorrect use of the device, incorrect procedures, or in case of use by inexperienced personnel.
- The procedure outlined above is a simple guide: medical specialists should use the device on the basis of their own experience.
- This is a sterile, non-pyrogenic device: it must not be re-used, re-sterilised, or re-conditioned in any way.
- Disposable single-use/single-patient: re-use of the disposable may cause cross infection to patient and/or user
- Before use, verify that the packaging is intact and check the validity period of the product.
- Do not use if the packaging is open, damaged or wet.
- Keep in a cool, dry place, away from heat and light sources.

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