

# Embryo Safe

Non-toxic Disinfectant for IVF Laboratory Floor

Ready to use solution

- **MEA & SMA Tested**
- Odourless
- Transparent
- Alcohol free
- VOC free
- Aldehyde free
- Phenol free
- Ammonium free
- Non allergic



- Non toxic
- Non corrosive
- Bactericidal
- Virucidal
- Fungisidal
- Sporicidal
- Neither medical nor chemical waste
- Fast acting
- Ecologic



ESSF 005

Manufactured by IVF Prime Inc.



Çamlıca Mahallesi Anadolu Bulvarı No: 20  
Timko Sokak N Blok No: 1 Yenimahalle  
Ankara / TURKEY  
info@ivfprime.com www.ivfprime.com

Lot no:

Manufacturing Date

Expiry Date


**5 Liters**

**Usage Areas:** It can be used in IVF and Andrology laboratories, in operating rooms and for disinfection of floor and surfaces.

**Usage:** Embryo Safe is poured in a container as needed. Application surface is wiped with an Embryo Safe sprayed mop or cloth and wait for 30 seconds. After 30 seconds, disinfection is achieved. If mechanical sterilization is needed, it should be waited for 5 minutes after wiping the surface with an Embryo Safe sprayed mop or cloth. Mechanical sterilization is achieved after 5 minutes. Then surface is being dried.

**Warning:** Ready-to-use Embryo Safe solution should not be diluted. It should not be blended with any chemical. It should not be drunk. It should not be exposed to direct sunlight and high temperature. It should be preserved at room temperature.

**Environmental Impacts:** Ingredients of Embryo Safe are not hazardous to nature and they can be dissolved naturally.

**Test Organism and Effect Time:** Staphylococcus aureus, Escherichia coli, Enterococcus faecalis, Pseudomonas aeruginosa MRSA SNUH 7135, HRSA SNUH 7191, MS-SA, MS-CNS, Acinetobacter Baumannii, Serratia marcescens, Proteus mircescens, Salmonella sp. Group A, Salmonella sp. Group B, Shigella S Bacillus cereus exneri, Shigella boydii, Shigella sonnei, Bacillus alvei, Bacillus circulans, Candida albicans, Candida tropicalis, Mycobacterium tuberculosis, Mycobacterium chelonae Helicobacter pylori, Hepa virus B, in f virus APR 8, HSV-HF, HIV-I, SARS

Embryo Safe effective against above listed bacteria, spore forming bacteria, fungi and viruses in 30 seconds. Microbiological effect is enabled in 30 seconds.

**Active Ingredient:** Hypochlorous Acid (HOCl)

**Expiry Date:** 2 years from the manufacturing date if unopened. 6 months after opening.

**pH:** 4 – 6

Manufactured by IVF Prime Inc.



5 Liters

### Dung dịch khử khuẩn lau sàn an toàn cho phôi

Mã sản phẩm: **ESSF005**

Số lưu hành:

Quy cách: 5 L/ chai

Số lô/ Hạn sử dụng: Xem trên bao bì sản phẩm

Bảo quản: Nhiệt độ phòng

HDSD: Xem trong bao bì

Xuất xứ: Thổ Nhĩ Kỳ

Chủ sở hữu số lưu hành: Công ty TNHH Toàn Ánh

Địa chỉ: 162 Cô Giang, phường Cô Giang, Q.1, TP. HCM

*Các thông tin khác, đề nghị xem trong tờ hướng dẫn sử dụng kèm theo*

Nhà sản xuất: IVF Prime Sağlık Urunleri San ve Tic AS

Đại chỉ: Camlica Mah Anadolu Bulvarı No: 20 Timko Sokak N1 Blok 06200

Yenimahalle Ankara Thổ Nhĩ Kỳ

**ĐẠI DIỆN HỢP PHÁP CÔNG TY**