

Human care handle bed FBH-730/720

Instruction manual

⚠ Caution

○ For safety, be sure to read the "Safety Guide" before use.
Accidents may damage the bed.





1. How to use the handle

- ① Handles are lined up on the foot side of the bed body. (Refer graph 1)
- ② Pull out only one handle to be used (① in Fig.2), put the grip (② in Fig.2)
- ③ Turn right to raise. Turn to the left to lower (See Fig.3)
- ④ It is the end point when the feel of turning the handle becomes heavy. After use, put back the handle and retract the handle. If the handle is in a position where it can't be stored, turn it in the opposite direction and store it.

⚠ Caution

- If the feel of turning the handle becomes heavy, don't force it.
Risk of injury due to damage.
- Don't rotate with other handles. The handle may be narrow
due to the narrow space between the handles.

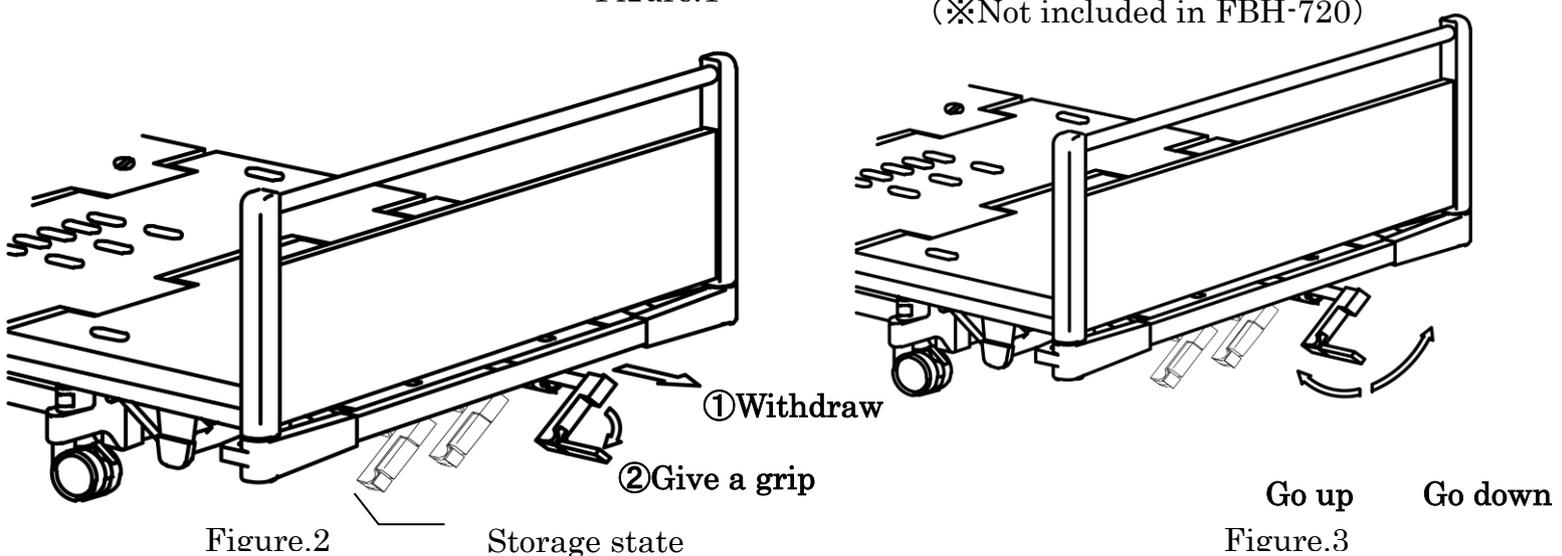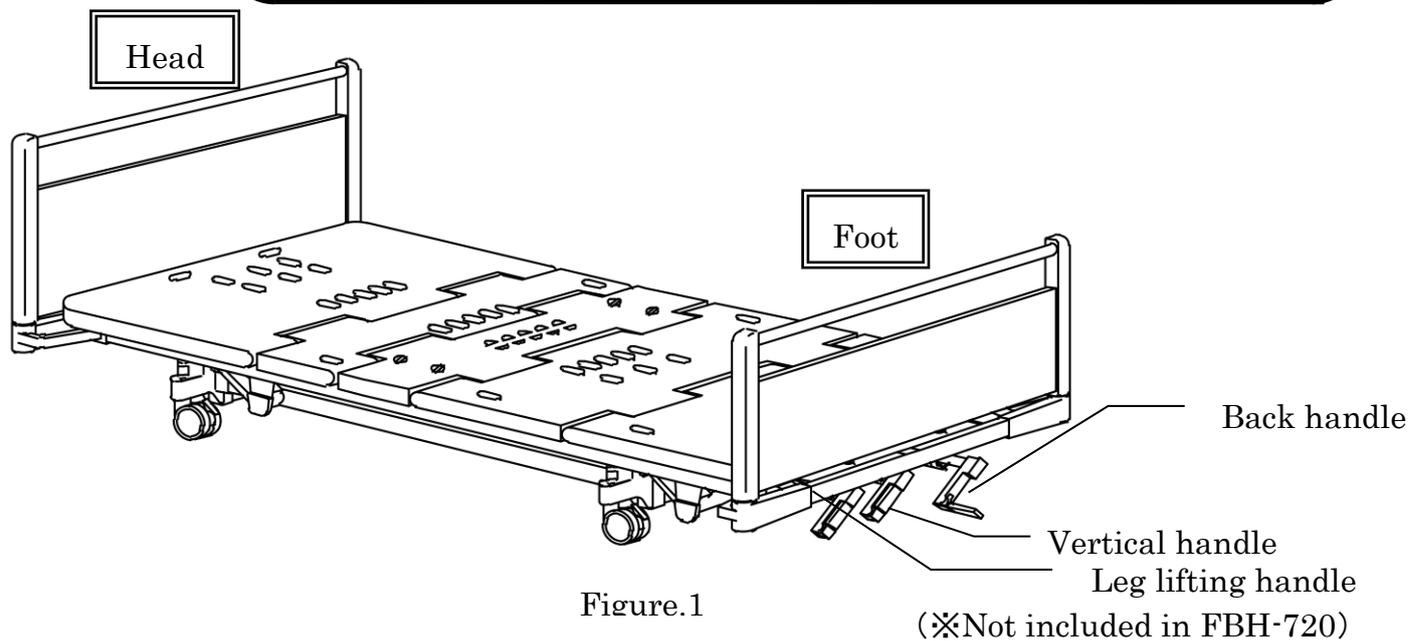

⚠ Caution

- Lock the casters when not moving. To fail to do so, it may move unexpectedly and cause an accident.



2. How to operate the switching lever (Dedicated for FBH-720)

By operating the switching lever, when you raise your back, you can switch between only raising your back and interlocking with raising your legs.

*** Be sure to perform the operation when the floorboard is horizontal.** The lever may not move, injury or damage to the bed may occur.

*** Don't use the switching lever at an angle.** (See Fig.4)

- When the lever is horizontal as shown in the figure, the leg lift moves in conjunction.

- When the lever is horizontal as shown in the figure, the leg lift moves in conjunction.


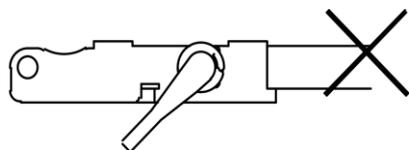


Figure.4

Back raising and leg raising are linked.

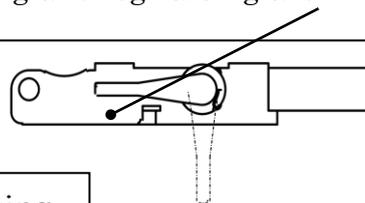

Only raising

Figure.5