

## HOW TO USE HIGH FLOW HUMIDIFIER ACCESSORIES

Secure the unit by mounting it on the rear of the machine in a top-down direction to the push-cart clamp.

Press and hold the lock where the humidifier is placed, place the humidifier on the heating plate with the air inlet facing forward, slide the humidifier forward and burn the heater with the air end facing forward, slide the humidifier tank forward and the latch closes when the humidifier is fully inserted.

Plug the infusion line into the bag of distilled water, making sure the Pressure Relief Valve is open.

Replace the filter: Turn counter-clockwise to open the filter cover, and place a new filter in the cavity. Turn clockwise to close and secure the filter assembly in place.

Connect the breathing wire with the built-in electric heating wire to the humidifier.

Connect the temperature sensor and heating cable: Connect the connection cable at the port of the compatible color on the unit; attach the 6-pin plug with a breathing wire.

Select and attach the appropriate nose frame for each patient.

**GIÁM ĐỐC**



**Nguyễn Thị Huệ Chi**