

USER MANUAL

Attention: This manual concerns 140101 Bacteria Filter and 140201 Bacteria Filter – Elite products.

REF. NUMBER/NUMBERS: 140101, 140201

PURPOSE:

These filters are produced as Bacteria-virus filter. Bacteria-virus filters are usually attached to the anesthesia or ventilation device before the patient circuit is installed. After the filter, the patient circuit is installed. Harmful particles are trapped at a rate of 99.9% thanks to the filtration property and prevented from going into the respiratory tract.

CONTENT INFORMATION: Bacteria Filter

WARNINGS and PRECAUTIONS:

For single use only. **DO NOT REUSE, DO NOT SUBMIT TO PROCESS OR DO NOT STERILIZE.** Reuse, treatment or sterilization may impair the structural integrity of the device and / or cause an injury or illness to the patient due to a malfunction of the device. Reuse or treatment may create a risk of contamination for the device and / or an infection of the patient or a contamination of a patient or a doctor by another patient and thus may lead to the transmission of infectious diseases from one patient to another and further problems. The contamination of the device may result in illness of the patient and of the users due to an infection. This personalized product is designed for single patient use.

Before use, examine carefully the device to verify that its contents or its sterile packaging are not damaged during transport. **DO NOT USE** if damaged. Please return immediately the damaged product to the dealer or to Plasti-med.

CONTRAINDICATIONS: The filters should be replaced when dirty otherwise they should be changed every 48 hours maximum.

COMPLICATIONS:

Make sure the filters connections are correct and there is no air leak.

INSTRUCTIONS FOR USE:

1. Determine the inhalation or exhalation input of the device to which the bacteria-virus filter will be attached.
2. Attach in an appropriate way the bacteria-virus filter to the device input you have determined.
3. Make sure the bacteria-virus filter is correctly attached to the device.
4. After attaching the bacteria-virus filter to the device, place the respiratory or anesthesia circuit at the other end of the filter.
5. After a 24-hour use, remove the bacteria-virus filter from the device and change it to a new one.

DURATION OF USE: The product is for single use and used during the application. It should be changed every 24 hours maximum.

TARGET AUDIENCE: All patient group.

STORAGE CONDITIONS: Store it at room temperature.

DISPOSAL METHOD: Disposal procedure for products after use:

Used products should be immediately treated as medical waste and disposed of in accordance with the medical waste regulations as soon as they are used to remove any contamination risk.



Caution



Do not re-use



Catalogue Number



Date of Manufacture



Use-by date



Batch code



Do not use if package is damaged



Do not resterile



Sterilized using ethylene oxide



Notified Body Number