

## USER MANUAL

### SPIROMETER FILTER WITH ELIPTIC MOUTH-STERILE

**REF. NUMBER/NUMBERS: 141 102, 141 103**

**PURPOSE:** The Filter is a disposable bacterial/viral filter designed to prevent biological contaminants from entering pulmonary function testing equipment.

**CONTENT INFORMATION: SPIROMETER FILTER WITH ELIPTIC MOUTH**

#### SUGGESTED PROCEDURE:

1. Remove from bag.
2. Inspect for damage, and dispose immediately if damaged.
3. Connect securely to pulmonary equipment.
4. If included, connect mouthpiece to small opening on filter.
5. After use, dispose of properly

**WARNING:** The use of this filter and mouthpiece (if included) is limited to a single patient, single session. Not intended for reprocessing. It is not to be used with neonate patients.

Do not soak, rinse, wash, gas, or steam sterilize this product. Patient secretions and wetting with cleaning solutions or liquid detergents may degrade the resistance and effectiveness of this filter.

**CAUTION:** Federal law (USA) restricts this device to sale by or on the order of a physician.



Caution



Do not  
re-use



Catalogue  
Number



Date of  
Manufacture



Use-by  
date



Batch  
code



Do not use if  
package is  
damaged



Do not  
resterile



Sterilized using  
ethylene oxide



Notified Body  
Number