

Ningbo Greetmed Medical Instruments Co.,Ltd

Instruction For Use

Indications for use:

The Guedel airway is an oropharyngeal airway that is inserted into the patient's pharynx through the mouth to provide an airway. Airway lumens have a smooth contour for improved airflow and easy access to the suction cannula. Flexible, carefully professionally filed anatomical shape to minimize airway trauma.

Operation process:

1. Choose the correct size (the correct size for the patient is determined by the distance between the corner of the mouth and the trachea).
2. Lubricate and insert with initial curvature facing up and then rotate halfway to follow blade curvature.
3. In patients with dysphagia, it can also be introduced at the outset with its curvature along the curvature of the tongue.
4. Once inserted, it is very important to support the jaw to maintain airway patency.

Warning:

Read this manual carefully before use.

This product should be used by trained personnel.

Storage:

Storage and transport temperature: -20°C to +60°C (-4°F to 140°F).

Keep the device in a cool, dry place away from direct sunlight before use.

Expiry date: 5 years from the date of manufacture.

Handling:

The device does not contain any hazardous substances and can be disposed of with your regular household trash