

How to fill a dry hydro massage device

AQUAI

with water



Prepared by
Marek Urbanowicz
murbanowicz@meden.com.pl

1. Unpack the device. The attached cardboard box should contain additional elements such as: (In addition, a jack to make it easier to remove the air from the device)

- Headrest
- Hose for filling the water
- Desinfection fluid
- Nipples
- Gaskets
- Quick couplers



2. Remove the two-piece cover. The housing is held by two metal "butterflies" on the bottom of the device. They should be turned so that the housing falls on the plastic legs of the device.



3. Push the case towards the control panel and remove the two screws connecting the two halves of the cases.



4. Very gently move the casing in the opposite direction and on the other side also unscrew the two screws that connect the casing. Then put the covers aside in a safe place.



5. Unscrew the vent plug and remove the water filler hose from the bottom of the machine.



6. Prepare a water filling hose and arm it with quick couplings, nipples and gaskets. Then pour the disinfectant liquid into it and connect it to the water supply.



7. Open the water tap halfway so that the air in the device can escape freely.



8. Control the water level in the device. When the water reaches the level of the diaphragm, close the valve and turn on the device for a while to vent the entire water system in the device. Then the water level should be topped up to the level of the black seal.



9. Close the vent plug and shut off the water supply.



10. Disconnect all hoses, plug them with plugs and hide under the device. Then run the P1 program and check if the device has no leaks.



11. After thoroughly inspecting the device, fold the covers in the opposite direction to disassembly.