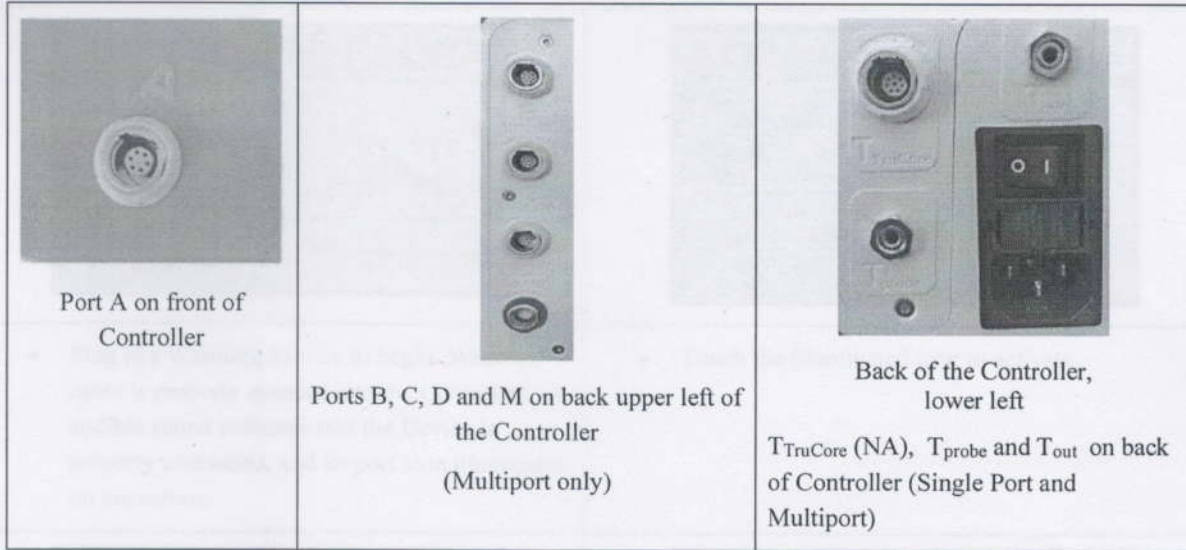


Figure 2: Controller Ports



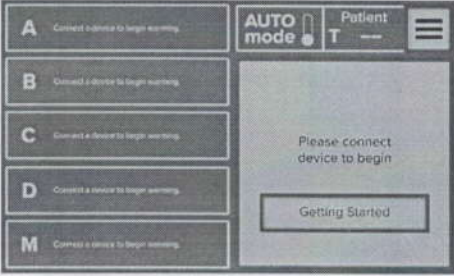
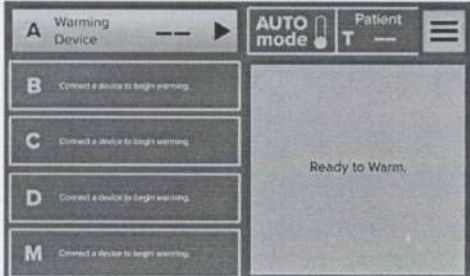
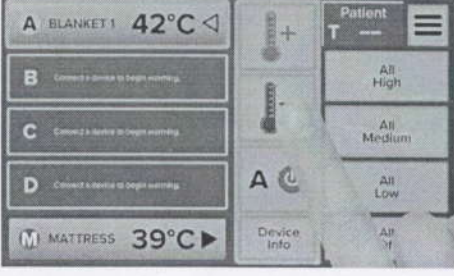
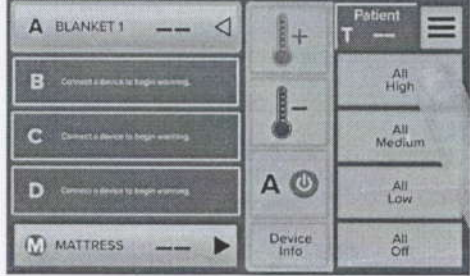
Note: When the connecting cable is inserted into the Controller, an audible sound indicates that the control sensor and over-temperature thermistor are present and functioning properly. The Port becomes available on the touchscreen.



Dặng Thanh Huy
GIÁM ĐỐC



4. Control Warming Devices using the touchscreen.

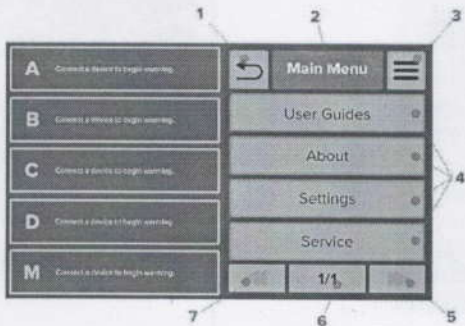
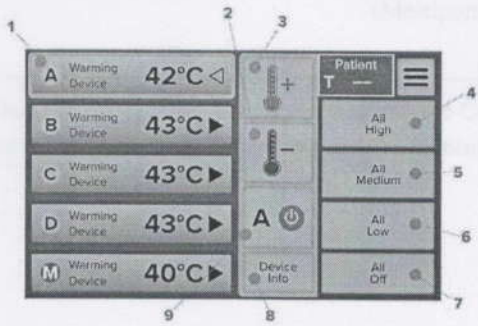
	
<ul style="list-style-type: none"> • Plug in a Warming Device to begin. When the cable is properly inserted into the Controller, an audible sound indicates that the Device is properly connected, and its port icon illuminates on the screen. 	<ul style="list-style-type: none"> • Touch the illuminated icon to activate.
	
<ul style="list-style-type: none"> • The Warming Device highlighted in green is the currently selected Device. To select its temperature, use the + thermometer to increase the temperature or the - thermometer to decrease the temperature. Turn off the highlighted port at the red power icon. 	<ul style="list-style-type: none"> • Each connected Warming Device can be adjusted separately. Other icons to the right adjust all port temperatures simultaneously: <ul style="list-style-type: none"> • All High • All Medium • All Low • All Off

5. When patient warming therapy is complete, turn the Mains Power Switch to OFF.

6. After use, disconnect the Controller from the electrical outlet.

3137
CÔ
T
HƯƠNG
MIỆ
ĐỨC

CONTROL SCREEN OVERVIEW

 <p>The image shows the 'Main Menu' screen. On the left, there are five rows of device status indicators labeled A, B, C, D, and M, each with a small icon and the text 'Connect a device to begin warming'. The main area contains a 'Main Menu' title at the top, followed by a list of menu items: 'User Guides', 'About', 'Settings', and 'Service'. At the bottom, there are navigation buttons for '1/1' and 'Previous Page'. Numbered callouts 1 through 7 point to specific UI elements: 1 (Back arrow), 2 (Main Menu title), 3 (Main Menu icon), 4 (Sub Menu items), 5 (Next Page arrow), 6 (Pages indicator), and 7 (Previous Page arrow).</p>	<ol style="list-style-type: none"> 1. Back 2. Menu Title 3. Main Menu 4. Sub Menu 5. Next Page 6. Pages 7. Previous Page
 <p>The image shows the 'Device Control' screen. On the left, there are five rows of device status indicators labeled A, B, C, D, and M, each showing 'Warming Device' and a temperature (42°C, 43°C, 43°C, 43°C, 40°C). The main area contains a 'Patient' title at the top, followed by a list of temperature settings: 'All High', 'All Medium', 'All Low', and 'All Off'. At the bottom, there are navigation buttons for 'Device Info' and 'Turn Off Selected Device'. Numbered callouts 1 through 9 point to specific UI elements: 1 (Currently Selected Device), 2 (Decrease Selected Device Temperature), 3 (Increase Selected Device Temperature), 4 (Set All Devices to High Temperature), 5 (Set All Devices to Medium Temperature), 6 (Set All Devices to Low Temperature), 7 (Turn Off All Devices), 8 (View Selected Device Info), and 9 (Turn off Selected Device).</p>	<ol style="list-style-type: none"> 1. Currently Selected Device (green border) 2. Decrease Selected Device Temperature 3. Increase Selected Device Temperature 4. Set All Devices to High Temperature 5. Set All Devices to Medium Temperature 6. Set All Devices to Low Temperature 7. Turn Off All Devices 8. View Selected Device Info 9. Turn off Selected Device
<p>Display Tips:</p> <ol style="list-style-type: none"> A. Do not press down hard. This is not needed. B. An overly thick glove may prevent detection of finger. C. Moisture on screen can confuse the system; ensure screen is dry before use. 	

