

## **TRACHEOSTOMY TUBE**

### **INSTRUCTION FOR USE:**

1. Peel off the package, and take out the tube.
2. Test inflate cuff prior to use (if any).
  1. Deflate cuff before intubation to aid instruction and avoid cuff damage.
  2. Inflate cuff using the minimum volume of air required to provide an effective seal.
3. After intubation, take out the introducer and connect the tube with the respiratory device.

### **CAUTION:**

1. Single use only.
2. Sterile if package is unopened or damaged.
3. Store in a cool dry place.