

Urea/BUN**Order information**

REF	CONTENT	Analyzer(s) on which cobas c pack(s) can be used
05171873 190*	Urea/BUN (1900 tests)	System-ID 05 6303 9 cobas c 701/702
05171873 214*	Urea/BUN (1900 tests)	System-ID 05 6303 9 cobas c 701/702

Materials required (but not provided):

10759350 190	Calibrator f.a.s. (12 x 3 mL)	Code 401
12149435 122	Precinorm U plus (10 x 3 mL)	Code 300
12149443 122	Precipath U plus (10 x 3 mL)	Code 301
05117003 190	PreciControl ClinChem Multi 1 (20 x 5 mL)	Code 391
05947626 190	PreciControl ClinChem Multi 1 (4 x 5 mL)	Code 391
05117216 190	PreciControl ClinChem Multi 2 (20 x 5 mL)	Code 392
05947774 190	PreciControl ClinChem Multi 2 (4 x 5 mL)	Code 392
05172152 190	Diluent NaCl 9 % (119 mL)	System-ID 08 6869 3

* Some kits shown may not be available in all countries.

English**System information**

UREAL: ACN 8418 (serum, plasma)

U-BUN: ACN 8421 (serum, plasma)

URELU: ACN 8417 (urine)

UBUNU: ACN 8428 (urine)

SUREA: ACN 8419 (STAT, reaction time: 5, serum, plasma)

SUBUN: ACN 8427 (STAT, reaction time: 5, serum, plasma)

SUREU: ACN 8420 (STAT, reaction time: 5, urine)

SBUNU: ACN 8429 (STAT, reaction time: 5, urine)

Intended use

In vitro test for the quantitative determination of urea/urea nitrogen in human serum, plasma and urine on Roche/Hitachi **cobas c** systems.

Summary¹

Urea is the major end product of protein nitrogen metabolism. It is synthesized by the urea cycle in the liver from ammonia which is produced by amino acid deamination. Urea is excreted mostly by the kidneys but minimal amounts are also excreted in sweat and degraded in the intestines by bacterial action.

Determination of blood urea nitrogen is the most widely used screening test for renal function. When used in conjunction with serum creatinine determinations it can aid in the differential diagnosis of the three types of azotemia: prerenal, renal and postrenal.

Elevations in blood urea nitrogen concentration are seen in inadequate renal perfusion, shock, diminished blood volume (prerenal causes), chronic nephritis, nephrosclerosis, tubular necrosis, glomerular nephritis (renal causes) and urinary tract obstruction (postrenal causes). Transient elevations may also be seen during periods of high protein intake. Unpredictable levels occur with liver diseases.

Test principle

Kinetic test with urease and glutamate dehydrogenase.^{2,3,4,5}

Urea is hydrolyzed by urease to form ammonium and carbonate.



In the second reaction 2-oxoglutarate reacts with ammonium in the presence of glutamate dehydrogenase (GLDH) and the coenzyme NADH to produce L-glutamate. In this reaction two moles of NADH are oxidized to NAD⁺ for each mole of urea hydrolyzed.



The rate of decrease in the NADH concentration is directly proportional to the urea concentration in the specimen and is measured photometrically.

Reagents - working solutions

R1 NaCl 9 %

R3 (STAT R2) TRIS buffer: 220 mmol/L, pH 8.6; 2-oxoglutarate: 73 mmol/L; NADH: 2.5 mmol/L; ADP: 6.5 mmol/L; urease (jack bean): ≥ 300 μkat/L; GLDH (bovine liver): ≥ 80 μkat/L; preservative; nonreactive stabilizers

R1 is in position C and R3 (STAT R2) is in position B.

Precautions and warnings

For in vitro diagnostic use for health care professionals. Exercise the normal precautions required for handling all laboratory reagents.

Infectious or microbial waste:

Warning: handle waste as potentially biohazardous material. Dispose of waste according to accepted laboratory instructions and procedures.

Environmental hazards:

Apply all relevant local disposal regulations to determine the safe disposal.

Safety data sheet available for professional user on request.

Reagent handling

Ready for use

Storage and stability**UREAL**

Shelf life at 2-8 °C: See expiration date on **cobas c** pack label.

On-board in use and refrigerated on the analyzer: 4 weeks

On-board on the Reagent Manager: 24 hours

Diluent NaCl 9 %

Shelf life at 2-8 °C: See expiration date on **cobas c** pack label.

On-board in use and refrigerated on the analyzer: 4 weeks

On-board on the Reagent Manager: 24 hours

Specimen collection and preparation

For specimen collection and preparation only use suitable tubes or collection containers.

Only the specimens listed below were tested and found acceptable.

Serum

Plasma: Li-heparin and K₂-EDTA plasma. Do not use ammonium heparin.

The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. When processing samples in primary

tubes (sample collection systems), follow the instructions of the tube manufacturer.

Urine

Bacterial growth in the specimen and high atmospheric ammonia concentrations as well as contamination by ammonium ions may cause erroneously elevated results.

Stability in <i>serum/plasma</i> . ⁶	7 days at 15-25 °C
	7 days at 2-8 °C
	1 year at (-15)-(-25) °C

Centrifuge samples containing precipitates before performing the assay. See the limitations and interferences section for details about possible sample interferences.

Stability in <i>urine</i> . ⁶	2 days at 15-25 °C
	7 days at 2-8 °C
	1 month at (-15)-(-25) °C

Materials provided

See "Reagents – working solutions" section for reagents.

Materials required (but not provided)

See "Order information" section

General laboratory equipment

Assay

For optimum performance of the assay follow the directions given in this document for the analyzer concerned. Refer to the appropriate operator's manual for analyzer-specific assay instructions.

The performance of applications not validated by Roche is not warranted and must be defined by the user.

Application for serum and plasma**cobas c 701/702 test definition**

Assay type	Rate A		
Reaction time / Assay points	10/23-29 (STAT 5/11-17)		
Wavelength (sub/main)	700/340 nm		
Reaction direction	Decrease		
Units	mmol/L (mg/dL, g/L)		
Reagent pipetting	Diluent (H ₂ O)		
R1	10 µL	90 µL	
R3 (STAT R2)	38 µL	108 µL	
<i>Sample volumes</i>	<i>Sample</i>	<i>Sample dilution</i>	
		<i>Sample</i>	<i>Diluent (NaCl)</i>
Normal	2 µL	–	–
Decreased	4 µL	20 µL	100 µL
Increased	4 µL	–	–

Application for urine**cobas c 701/702 test definition**

Assay type	Rate A		
Reaction time / Assay points	10/23-29 (STAT 5/11-17)		
Wavelength (sub/main)	700/340 nm		
Reaction direction	Decrease		
Units	mmol/L (mg/dL, g/L)		
Reagent pipetting	Diluent (H ₂ O)		
R1	10 µL	90 µL	
R3 (STAT R2)	38 µL	108 µL	
<i>Sample volumes</i>	<i>Sample</i>	<i>Sample dilution</i>	
		<i>Sample</i>	<i>Diluent (NaCl)</i>

Normal	2 µL	3 µL	147 µL
Decreased	2 µL	2 µL	178 µL
Increased	2 µL	–	–

Calibration

Calibrators	S1: H ₂ O
	S2: C.f.a.s.
Calibration mode	Linear
Calibration frequency	2-point calibration
	- after reagent lot change
	- as required following quality control procedures

Calibration interval may be extended based on acceptable verification of calibration by the laboratory.

Traceability: This method has been standardized against ID/MS.

Quality control*Serum/plasma*

For quality control, use control materials as listed in the "Order Information" section. In addition, other suitable control material can be used.

Urine

Quantitative urine controls are recommended for routine quality control.

The control intervals and limits should be adapted to each laboratory's individual requirements. Values obtained should fall within the defined limits. Each laboratory should establish corrective measures to be taken if values fall outside the defined limits.

Follow the applicable government regulations and local guidelines for quality control.

Calculation

cobas c systems automatically calculate the analyte concentration of each sample.

Conversion factors:	mmol/L urea x 6.006 = mg/dL urea
	mmol/L urea x 0.06006 = g/L urea
	mmol/L urea nitrogen x 2.801 = mg/dL urea nitrogen
	mmol/L urea nitrogen x 0.02801 = g/L urea nitrogen
	mg/dL urea x 0.467 = mg/dL urea nitrogen

When 24-hour urine is used as the specimen, multiply the result by the 24-hour volume to obtain values in g or mmol/24 hours.

Limitations - interference

Criterion: Recovery within ± 10 % of initial value at a urea concentration of 8.3 mmol/L (49.8 mg/dL urea, 23.2 mg/dL urea nitrogen) in serum/plasma and at a urea concentration of 150 mmol/L (901 mg/dL urea, 421 mg/dL urea nitrogen) in urine. Recovery within ± 10 % for drug interference.

Serum/plasma

Icterus:⁷ No significant interference up to an I index of 60 (approximate conjugated and unconjugated bilirubin concentration: 1026 µmol/L or 60 mg/dL).

Hemolysis:⁷ No significant interference up to an H index of 1000 (approximate hemoglobin concentration: 621 µmol/L or 1000 mg/dL).

Lipemia (Intralipid):⁷ No significant interference up to an L index of 1000. There is poor correlation between the L index (corresponds to turbidity) and triglycerides concentration.

Ammonium ions may cause erroneously elevated results.

Drugs: No interference was found at therapeutic concentrations using common drug panels.^{8,9}

In very rare cases, gammopathy, in particular type IgM (Waldenström's macroglobulinemia), may cause unreliable results.¹⁰

Urine

Drugs: No interference was found at therapeutic concentrations using common drug panels.⁹

For diagnostic purposes, the results should always be assessed in conjunction with the patient's medical history, clinical examination and other findings.

ACTION REQUIRED

Special Wash Programming: The use of special wash steps is mandatory when certain test combinations are run together on **cobas c** systems. All special wash programming necessary for avoiding carry-over is available via the **cobas** link, manual input is required in certain cases. The latest version of the carry-over evasion list can be found with the NaOHD/SMS/SmpCln1+2/SCCS Method Sheet and for further instructions refer to the operator's manual.

Where required, special wash/carry-over evasion programming must be implemented prior to reporting results with this test.

Limits and ranges

Measuring range

Serum/plasma

0.5-40 mmol/L (3.0-240 mg/dL urea, 1.4-112 mg/dL urea nitrogen)

Determine samples having higher concentrations via the rerun function. Dilution of samples via the rerun function is a 1:3 dilution. Results from samples diluted using the rerun function are automatically multiplied by a factor of 3.

Urine

1-2000 mmol/L (6-12000 mg/dL urea, 2.8-5600 mg/dL urea nitrogen)

Determine samples having higher concentrations via the rerun function. Dilution of samples via the rerun function is a 1:1.8 dilution. Results from samples diluted using the rerun function are automatically multiplied by a factor of 1.8.

Determine samples having concentrations lower than the technical limit of 40 mmol/L (240 mg/dL urea and 112 mg/dL urea nitrogen) via the rerun function. Samples are measured undiluted.

Lower limits of measurement

Lower detection limit of the test

Serum/plasma

0.5 mmol/L (3.0 mg/dL urea, 1.4 mg/dL urea nitrogen)

The lower detection limit represents the lowest measurable analyte level that can be distinguished from zero. It is calculated as the value lying 3 standard deviations above that of the lowest standard (standard 1 + 3 SD, repeatability, n = 21).

Values below the lower detection limit (< 0.5 mmol/L) will not be flagged by the instrument.

Urine

1 mmol/L (6 mg/dL urea, 2.8 mg/dL urea nitrogen)

The lower detection limit represents the lowest measurable analyte level that can be distinguished from zero. It is calculated as the value lying 3 standard deviations above that of the lowest standard (standard 1 + 3 SD, repeatability, n = 21).

Values below the lower detection limit (< 1 mmol/L) will not be flagged by the instrument.

Expected values

Urea:

Serum/plasma¹¹

Adults 2.76-8.07 mmol/L (16.6-48.5 mg/dL)

Urine

24-hour urine¹² 428-714 mmol/24 h (25.7-42.9 g/24 h), corresponding to 286-595 mmol/L (1.71-3.57 g/dL)^{a)}

a) Based on an average urine output of 1.2-1.5 L/24 h

Urea nitrogen (BUN):

Serum/plasma¹²

Adults (18-60 years) 2.14-7.14 mmol/L 6-20 mg/dL

Adults (60-90 years) 2.86-8.21 mmol/L 8-23 mg/dL

Infants (< 1 year) 1.43-6.78 mmol/L 4-19 mg/dL

Infants/children 1.79-6.43 mmol/L 5-18 mg/dL

Urine

24-hour urine¹² 428-714 mmol/24 h (12-20 g/24 h), corresponding to 286-595 mmol/L (801-1666 mg/dL)^{b)}

b) Based on an average urine output of 1.2-1.5 L/24 h

Each laboratory should investigate the transferability of the expected values to its own patient population and if necessary determine its own reference ranges.

Specific performance data

Representative performance data on the analyzers are given below. Results obtained in individual laboratories may differ.

Precision

Precision was determined using human samples and controls in an internal protocol. *Serum/plasma:* Repeatability (n = 21) and intermediate precision (3 aliquots per run, 1 run per day, 21 days); *Urine:* Repeatability (n = 21) and intermediate precision (3 aliquots per run, 1 run per day, 10 days). The following results were obtained:

Serum/plasma

UREAL/U-BUN:

Repeatability	Mean mmol/L (mg/dL urea)	SD mmol/L (mg/dL urea)	CV %
Precinorm U	7.10 (42.6)	0.07 (0.4)	0.9
Precipath U	26.3 (158)	0.3 (2)	1.1
Human serum A	7.20 (43.2)	0.09 (0.5)	1.2
Human serum B	16.4 (98.5)	0.1 (0.6)	0.7
Human serum C	35.1 (210)	0.3 (2)	0.7

SUREA/SUBUN:

Repeatability	Mean mmol/L (mg/dL urea)	SD mmol/L (mg/dL urea)	CV %
Precinorm U	7.00 (42.0)	0.06 (0.4)	0.9
Precipath U	26.2 (157)	0.2 (1)	0.9
Human serum A	7.10 (42.6)	0.07 (0.4)	1.0
Human serum B	16.4 (98.5)	0.1 (0.6)	0.8
Human serum C	35.0 (210)	0.2 (1)	0.6

UREAL/U-BUN + SUREA/SUBUN:

Intermediate precision	Mean mmol/L (mg/dL urea)	SD mmol/L (mg/dL urea)	CV %
Precinorm U	6.66 (40.0)	0.08 (0.5)	1.2
Precipath U	23.2 (139)	0.3 (2)	1.1
Human serum 3	9.13 (54.8)	0.10 (0.6)	1.1
Human serum 4	14.9 (89.5)	0.2 (1.2)	1.3

Urine

URELU/UBUNU:

Repeatability	Mean mmol/L (mg/dL urea)	SD mmol/L (mg/dL urea)	CV %
Control level 1	154 (925)	2 (12)	1.4
Control level 2	250 (1501)	2 (12)	1.0
Human urine A	110 (661)	2 (12)	2.0
Human urine B	350 (2102)	3 (18)	0.8

Urea/BUN

Human urine C 1877 (11273) 15 (90) 0.8

SUREU/SBUNU:

Repeatability	Mean mmol/L (mg/dL urea)	SD mmol/L (mg/dL urea)	CV %
Control level 1	148 (889)	3 (18)	1.9
Control level 2	246 (1477)	3 (18)	1.2
Human urine A	107 (643)	2 (12)	1.5
Human urine B	345 (2072)	2 (12)	0.7
Human urine C	1875 (11261)	13 (78)	0.7

URELU/UBUNU + SUREU/SBUNU:

Intermediate precision	Mean mmol/L (mg/dL urea)	SD mmol/L (mg/dL urea)	CV %
Control level 1	154 (925)	4 (24)	2.7
Control level 2	280 (1682)	6 (36)	2.3
Human urine 3	316 (1898)	6 (36)	2.0
Human urine 4	133 (799)	3 (18)	2.4

Results for intermediate precision were obtained on the master system **cobas c 501** analyzer.

Method comparison

Urea values for human serum, plasma and urine samples obtained on a **cobas c 701** analyzer (y) were compared with those determined on **cobas c 501** analyzer (x), using the corresponding reagent.

*Serum/plasma***UREAL/U-BUN:**

Sample size (n) = 114

Passing/Bablok ¹³	Linear regression
$y = 1.000x + 0.000$ mmol/L	$y = 1.004x - 0.077$ mmol/L
$\tau = 0.989$	$r = 0.999$

The sample concentrations were between 3.10 and 39.6 mmol/L (18.6 and 238 mg/dL urea).

SUREA/SUBUN:

Sample size (n) = 114

Passing/Bablok ¹³	Linear regression
$y = 1.000x + 0.10$ mmol/L	$y = 1.004x + 0.09$ mmol/L
$\tau = 0.984$	$r = 0.999$

The sample concentrations were between 2.9 and 39.5 mmol/L (17.4 and 237 mg/dL urea).

*Urine***URELU/UBUNU:**

Sample size (n) = 134

Passing/Bablok ¹³	Linear regression
$y = 0.983x - 2.55$ mmol/L	$y = 0.988x - 5.15$ mmol/L
$\tau = 0.977$	$r = 1.000$

The sample concentrations were between 11.7 and 1995 mmol/L (70.3 and 11982 mg/dL urea).

SUREU/SBUNU:

Sample size (n) = 135

Passing/Bablok ¹³	Linear regression
$y = 0.987x - 5.01$ mmol/L	$y = 1.021x - 19.99$ mmol/L
$\tau = 0.973$	$r = 0.999$

The sample concentrations were between 11.0 and 1965 mmol/L (66.1 and 11802 mg/dL urea).

References

- 1 Rock RC, Walker WG, Jennings CD. Nitrogen metabolites and renal function. In: Tietz NW, ed. *Fundamentals of Clinical Chemistry*. 3rd ed. Philadelphia: WB Saunders 1987;669–704.
- 2 Richterich R, Colombo JP. *Klinische Chemie*. 4th ed. Basel: Karger S 1978;319-324.
- 3 Talke H, Schubert GA. Enzymatische Harnstoffbestimmung in Blut und Serum im optischen Test nach Warburg. *Klin Wochenschr* 1965;43:174.
- 4 Tiffany TO, Jansen JM, Burtis CA, et al. Enzymatic kinetic rate and end-point analyses of substrate, by use of a GeMSAEC Fast Analyzer. *Clin Chem* 1972;18:829-840.
- 5 Sampson EJ, Baired MA, Burtis CA, et al. A coupled-enzyme equilibrium method for measuring urea in serum: Optimization and evaluation of the AACC study group on urea candidate reference method. *Clin Chem* 1980;26:816-826.
- 6 WHO Publication: Use of anticoagulants in diagnostic laboratory investigations, WHO/DIL/LAB/99.1 Rev.2:Jan 2002.
- 7 Glick MR, Ryder KW, Jackson SA. Graphical Comparisons of Interferences in Clinical Chemistry Instrumentation. *Clin Chem* 1986;32:470-475.
- 8 Breuer J. Report on the Symposium "Drug effects in Clinical Chemistry Methods". *Eur J Clin Chem Clin Biochem* 1996;34:385-386.
- 9 Sonntag O, Scholer A. Drug interference in clinical chemistry: recommendation of drugs and their concentrations to be used in drug interference studies. *Ann Clin Biochem* 2001;38:376-385.
- 10 Bakker AJ, Mücke M. Gammopathy interference in clinical chemistry assays: mechanisms, detection and prevention. *Clin Chem Lab Med* 2007;45(9):1240-1243.
- 11 Löhr B, El-Samalouti V, Junge W, et al. Reference Range Study for Various Parameters on Roche Clinical Chemistry Analyzers. *Clin Lab* 2009;55:465-471.
- 12 Wu AHB, ed. *Tietz Clinical Guide to Laboratory Tests*, 4th edition. St. Louis (MO): Saunders Elsevier 2006;1096.
- 13 Bablok W, Passing H, Bender R, et al. A general regression procedure for method transformation. Application of linear regression procedures for method comparison studies in clinical chemistry, Part III. *J Clin Chem Clin Biochem* 1988 Nov;26(11):783-790.

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard (for USA: see dialog.roche.com for definition of symbols used):

	Contents of kit
	Volume after reconstitution or mixing
	Global Trade Item Number

COBAS, COBAS C, PRECINORM, PRECIPATH and PRECICONTROL are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2021, Roche Diagnostics

0105171873190c701V11.0

UREAL

Urea/BUN

CE 0123



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

+800 5505 6606



cobas®